nenob[®]

Leite




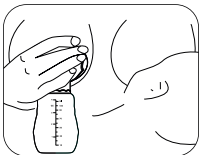
Table of contents

Spis treści

4	Instrukcja obsługi	PL
5	User manual	EN
7	Bedienungsanleitung	DE
9	Návod k použití	CZ
10	Használati utasítás	HU
12	Používateľská príručka	SK
14	Användarhandbok	SE
15	Käyttäjän käsikirja	FI
17	Brukerhåndbok	NO
19	Brugermanual	DK
20	Gebruikershandleiding	NL
22	Manual del usuario	ES
24	Manuale utente	IT
25	Manuel de l'utilisateur	FR
27	Manual de utilizare	RO
29	Manual do utilizador	PT
30	Korisnički priručnik	HR/BA
32	Korisničko uputstvo	RS/ME
33	Uporabniški priročnik	SI
35	Εγχειρίδιο χρήστη	GR
37	Naudotojo vadovas	LT
38	Lietotāja rokasgrāmata	LV
40	Kasutusjuhend	ET
41	Інструкція користувача	UA
43	Упатство за корисник	MK
45	Ръководство за потребителя	BG

A

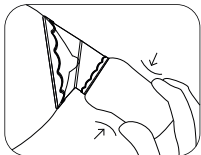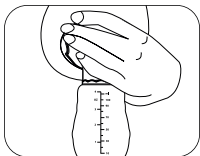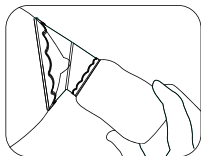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Szanowny Kliencie,

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EN

USER MANUAL

Dear Customer,
thank you for purchasing Neno Leite, a silicone milk collector for collecting milk during feeding/pumping from the other breast. It aims to collect food in a simple and effective way.

01. SAFETY MEASURES

1. Before each use, check the condition of the product.

2. We recommend using only soft brushes or sponges for cleaning, hard materials may scratch.
3. If there is any sign of damage, replace the product.
4. Do not store near sharp objects.
5. This product is not a toy. The collector should be used in accordance with its intended use.
6. Store the product in a dry place and avoid direct sunlight.

02. PRODUCT FEATURES

Easy to use

With just one squeeze, the milk collector draws out your milk with the power of continuous suction-no noise, no electricity and no cords.

BPA FREE Material

Made of high quality of silicone. No BPA, PCV and phthalate.

Painless and work silent

The milk collector helps to express breast milk discreetly, silently and quickly. Avoid embarrassment in public places.

No Leak Any More

When most mother are breastfeeding, the breast on the other side would often spill breast milk which may wet your clothes or your baby, or be wasted on the disposable nursing pad. The milk collector can collect every drop of breast milk while you're breastfeeding.

Easy to clean

All-in-one design and no disassemble required. They are very easy to clean by clear boiling water for about 2 minutes.

03. HOW TO USE

SEE FIG. A

1. Make sure your milk collector has been cleaned before and after use.
2. Squeeze the pump to push internal air out.
3. Invert the flange, place it onto your nipple centered in the middle of the breast shield.
4. Restore the flange, then the pump would firmly suck onto your breast.
5. You may use the product as a collector. The milk will then flow freely to the pump. If you want to use it as a manual breast pump - squeeze the pump to facilitate milk leak.

04. SPECIFICATION

Material: Silicone+PP

Capacity: 100 ml

05. WARRANTY CARD

The product is covered by a 24-month warranty.

The terms of the guarantee can be found at: <https://nenop.pl/gwarancja>

Details, contact and website address can be found at: <https://nenop.pl/kontakt>

Specifications and contents of the kit are subject to change without notice.

We apologise for any inconvenience.

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