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Warm application:

- Abdominal pain • Colic • Constipation • Cramps and muscle overcooling • menstrual pain

How to heat a hot water bottle?

- in a microwave with a power of 300-600 W max. 1 minute • in an oven preheated to 120°C, approx. 4 minutes • on the radiator until the hot water bottle is sufficiently heated and warm

The product is covered by a 2-year warranty. If you wish to file a complaint, please to send a message to the following address: info@kgktrend.pl

Distributor:

KGK TREND Sp. z o.o.

Ujastek 5b, 31-752

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www.neno.pl

**Manufacturer's data:**

Medi Partners Sp. z o.o.,

43-419 Hażlach,

Śloneczna 1

Made in Poland

Cold Application:

- lactation fullness • strokes • swelling • Fever • Headache • Insect bites

How to cool a hot water bottle?

- in the freezer or refrigerator • put in a clean and dry plastic bag, for about 30-60 minutes.

Specification

- Diameter: approx. 15 cm
- Weight: approx. 180 grams
- fabric: 100% cotton • filling: cherry seeds, perfectly cleaned, undamaged and from Polish crops
- in case of dirt, gently wipe with a damp cloth
- the maximum contact time of the hot water bottle with the skin (cold or warm) is 20 minutes
- it is natural to release the subtle scent of cherries when heated.

WARNING: If you experience any discomfort or irritation to your skin, stop using the product immediately and consult a doctor.

Batch number: TM142

KOD Basic UDI-DI: 590303142TermoforMDS8

A product in accordance with the Act of 7 April 2022 on medical devices

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