2 shell-shaped collectors for neno breast pumps  
2 muszle do laktatorów neno


# Table of contents

## Spis treści

|           |                                   |              |
|-----------|-----------------------------------|--------------|
| <b>8</b>  | <b>Instrukcja obsługi</b>         | <b>PL</b>    |
| <b>11</b> | <b>User manual</b>                | <b>EN</b>    |
| <b>14</b> | <b>Bedienungsanleitung</b>        | <b>DE</b>    |
| <b>18</b> | <b>Návod k použití</b>            | <b>CZ</b>    |
| <b>21</b> | <b>Használati utasítás</b>        | <b>HU</b>    |
| <b>25</b> | <b>Používateľská príručka</b>     | <b>SK</b>    |
| <b>28</b> | <b>Användarhandbok</b>            | <b>SE</b>    |
| <b>31</b> | <b>Käyttäjän käsikirja</b>        | <b>FI</b>    |
| <b>35</b> | <b>Brukerhåndbok</b>              | <b>NO</b>    |
| <b>38</b> | <b>Brugermanual</b>               | <b>DK</b>    |
| <b>41</b> | <b>Gebruikershandleiding</b>      | <b>NL</b>    |
| <b>45</b> | <b>Manual del usuario</b>         | <b>ES</b>    |
| <b>48</b> | <b>Manuale utente</b>             | <b>IT</b>    |
| <b>52</b> | <b>Manuel de l'utilisateur</b>    | <b>FR</b>    |
| <b>56</b> | <b>Manual de utilizare</b>        | <b>RO</b>    |
| <b>59</b> | <b>Manual do utilizador</b>       | <b>PT</b>    |
| <b>63</b> | <b>Korisnički priručnik</b>       | <b>HR/BA</b> |
| <b>66</b> | <b>Korisničko uputstvo</b>        | <b>RS/ME</b> |
| <b>69</b> | <b>Uporabniški priročnik</b>      | <b>SI</b>    |
| <b>72</b> | <b>Εγχειρίδιο χρήστη</b>          | <b>GR</b>    |
| <b>76</b> | <b>Naudotojo vadovas</b>          | <b>LT</b>    |
| <b>79</b> | <b>Lietotāja rokasgrāmata</b>     | <b>LV</b>    |
| <b>83</b> | <b>Kasutusjuhend</b>              | <b>ET</b>    |
| <b>86</b> | <b>Інструкція користувача</b>     | <b>UA</b>    |
| <b>90</b> | <b>Упатство за корисник</b>       | <b>MK</b>    |
| <b>93</b> | <b>Ръководство за потребителя</b> | <b>BG</b>    |


1

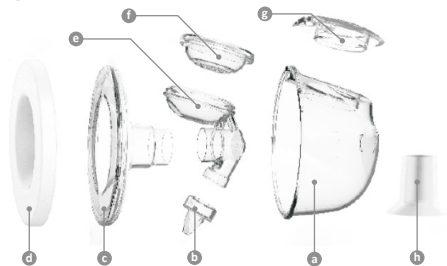2

3

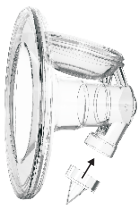
4

5

6

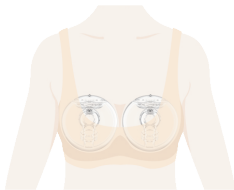✓ IDEAL

Upright

✓ GOOD

Slightly forward

✓ OK

Bending forward

✗ MAY LEAK

Reclining back

✗ WON'T WORK

Laying back (or side)

8

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# EN

## USER MANUAL

Dear Customer, The Neno breast pump shell kit you have purchased is a product for the Neno breast pump. This product is designed to be used with one of the Neno breast pumps to express breast milk from a woman's breast after giving birth. Please read the instruction manual before use.

### 01. PRECAUTIONS

1. The product is not heat resistant. Keep the device away from hot surfaces and open flames. Do not place the product in a hot oven.
2. Never use the product if you see damage or wear that prevents or impairs operation. Check the product before each use.
3. The product is intended for use by a single user.
4. Do not repair or modify the product yourself.
5. Use only with Neno brand breast pumps.
6. Before first use, wash and disinfect all parts of the product that come into direct contact with your breast or milk.
7. Wash all product components after each use.
8. When using the product around children, take special care.

9. Do not allow your child to play with the elements or the product.
10. Keep all components out of the reach of children.
11. Do not use abrasive or antibacterial agents when cleaning the product. Do not place product parts directly on surfaces that have been cleaned with antibacterial agents.
12. Too much concentration of hygiene products can cause the plastic to crack. In case of visible cracks in the product, discard it immediately.
13. Sterilisation and high temperatures can lead to distortion and the cap not fitting into the bottle. The maximum temperature for washing and sterilisation is 100°C, no longer than 5 minutes for silicone parts and no longer than 10 minutes for other parts.
14. Sterilising components in hard water can cause sedimentation.

## 02. CONTENTS OF THE KIT

2x 240ml container | 2x connectors | 2x membrane lid | 4x membrane | 4x valve  
 | 2x 24mm breast shields | 2x funnel lid | 2x 17 mm breast shield size reducer | 2x  
 19mm breast shield size reducer | 2x 21mm breast shield size reducer

## 03. BEFORE FIRST USE

1. Disassemble the shells into individual components. **SEE FIG.1**
  - a. 240 ml container
  - b. Valve
  - c. Breast shield
  - d. Breast shield lid
  - e. Connector
  - f. Membrane
  - g. Membrane cover
  - h. Funnel reducer
2. Wash the components in warm water with mild soap. Only use drinkable tap water for washing. Use bottled water if the water in your tap does not meet this requirement.
3. When cleaning the valve, do not use brushes or similar objects. Be careful not to puncture the valve on the way out.
4. Let the elements dry naturally .

## 04. CLEANING

1. Disassemble the shells into individual components. **SEE FIG.1**
2. Rinse in cold water just after use.
3. Wash the components in warm water with mild soap. Only use drinkable tap water for washing. Use bottled water if the water in your tap does not meet this requirement.
4. Rinse all components thoroughly.

5. Let the elements dry naturally.

## 05. DISINFECTION

1. Disinfect before first use and once a day.
2. Use the sterilisation device and select a suitable programme. The maximum temperature is 100°C, no longer than 5 minutes for silicone items and no longer than 10 minutes for other items.
3. You can also use a pot, boil the water to 100°C and turn off the heat. Put the parts in the pot and cover with a lid for 10 minutes to maintain the temperature as long as possible. Do not boil.
4. Sterilising components in hard water can cause sediment to form.

## 06. MONTAGE

1. Wash your hands thoroughly with soap and water before touching the shell.
2. Fit the membrane to the connector and then the breast shield. **SEE FIG.2**
3. Fit the valve to the connector, making sure it is in the correct position. **SEE FIG.3**
4. Insert the whole into the shell, paying particular attention to the part with the membrane. Then place the edges of the breast shield over the container. **SEE FIG.4**
5. Place the membrane cover over the membrane. **SEE FIG.5**
6. Notes:
  - a. Before each use, check the parts for damage. If necessary, replace damaged parts with new ones.
  - b. Always check the cleanliness of components before each assembly and use.

## 07. USE OF SHELLS

1. Wash your hands thoroughly with soap and water before touching the shell.
2. Insert the shells into the bra so that the shells have the correct positions and also so that the nipple is inside the drainage channel. **SEE FIG.6**
3. Connect the shells using the tubes from the Neno breast pump. If the tubes stand out too much in the bra and are uncomfortable then you can remove the plastic part of the tube and connect the silicone tube directly to the shell.
4. Turn on the breast pump by pressing the power button and select your preferred mode. When expressing, remember not to tilt your chest back as this may cause milk to leak out or prevent you from expressing milk. **SEE FIG.7**
5. Press the power button to switch off the breast pump after use.
6. Disconnect the breast pump from the shell by unhooking the tubing from the shell, then pour the expressed milk into the container. While pouring

the milk, squeeze the breast shield with your thumb to control the flow.  
**SEE FIG.8**

## 08. FAQ

- 1. Problem:** Condensation in tubes  
**Possible reason:** Ensure tubes, membranes and membrane covers are dry before assembly and use. Disconnect the tubes from the breast pump and make sure, before reconnecting, that the previously mentioned components are dry. Only then can you use the shells again.
- 2. Issue 2:** Milk in tubes  
**Possible reason:** Check membrane for damage. Replace membrane and replace tube if necessary.
- 3. Problem:** No or weak suction power  
**Possible reason:** Check the tube to make sure it is well inserted into the membrane cover.  
**Possible cause:** Ensure that the valves are in the correct position and check for damage or wear. If necessary, replace the valve.  
**Possible reason:** Check that all parts are correctly assembled.

## 09. SPECIFICATION

**Material of construction:** PP/Silicone | **BPA-free** | **Dimensions:** 120x120x74mm | **Weight:** 150g | **Capacity:** 240ml

## 10. WARRANTY CARD

The product comes with a 24-month warranty. Warranty terms and conditions can be found at: <https://neno.pl/gwarancja>.

Details, contact and service address can be found at: <https://neno.pl/kontakt>.

Specifications and contents are subject to change without notice. We apologise for any inconvenience.

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EN

The crossed out trash can symbol indicates that unusable electrical or electronic devices, its accessories (such as power supplies, cords) or components (for example batteries, if included) cannot be disposed of alongside with household waste. In order to dispose of the devices or its components (for example, batteries) deliver the device to the collection point, where it will be accepted free of charge. Disposal is subject to the recast version of the WEEE Directive (2012/19/ EU) and the Directive on batteries and accumulators (2006/66 / EC). Proper disposal of the device prevents degradation of the natural environment. Information about the collection points of the facilities is issued by the competent local authorities. Incorrect disposal of waste is subject to penalties provided for by the law in force in the given area.

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