nenor[®]

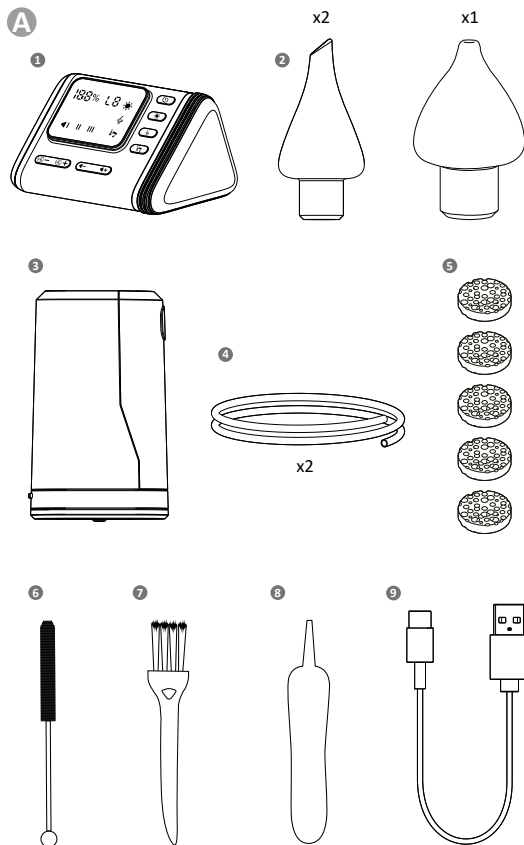
Nariz

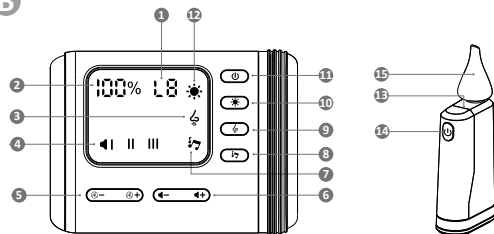
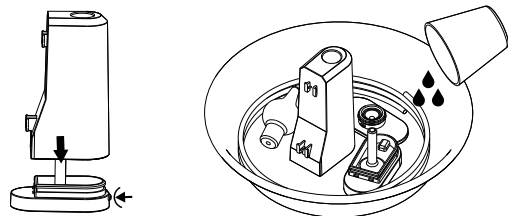
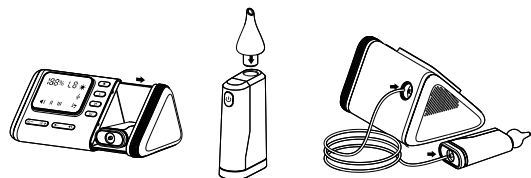
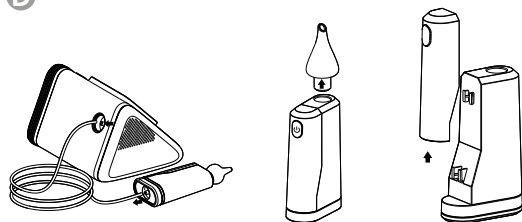
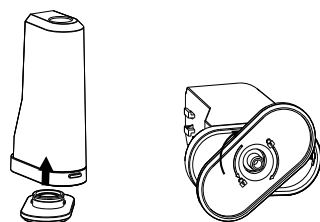
Table of contents

Spis treści

6	INSTRUKCJA OBSŁUGI	PL
9	USER MANUAL	EN
12	BEDIENUNGSANLEITUNG	DE
15	NÁVOD K POUŽITÍ	CZ
18	HASZNÁLATI UTASÍTÁS	HU
21	POUŽÍVATEĽSKÁ PRÍRUČKA	SK
24	ANVÄNDARHANDBOK	SE
27	KÄYTTÄJÄN KÄSIKIRJA	FI
30	BRUKERHÅNDBOK	NO
33	BRUGERMANUAL	DK
36	GEbruikersHANDLEIDING	NL
39	MANUAL DEL USUARIO	ES
42	MANUALE UTENTE	IT
46	MANUEL DE L'UTILISATEUR	FR
49	MANUAL DE UTILIZARE	RO
52	KORISNICKI PRIRUČNIK	HR/BA
55	UPUTSTVO ZA UPOTREBU	RS/ME
58	UPORABNIŠKI PRAVILNIK	SI
61	ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΤΗ	GR
64	LĪETOTĀJĀ ROKASGRĀMATA	LV
67	VARTOTOJO VADOVAS	LT
70	KASUTUSJUHEND	ET
73	ПОСІБНИК КОРИСТУВАЧА	UA
77	РЪКОВОДСТВО ЗА ПОТРЕБИТЕЛЯ	BG
80	УПАТСТВО ЗА УПОТРЕБА	MK
83	MANUAL DO UTILIZADOR	PT

B**E****C****F****D****G**

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USER MANUAL

Dear Customer,

Thank you for purchasing the Neno Nariz nasal aspirator. The product is used to suction secretions from a child's nose. The device is intended for children from birth to 6 years of age. This instruction manual contains all the information necessary for correct use. Please familiarise yourself with it and keep it in case you need to use it again.

01. PRECAUTIONS

1. Do not disassemble or repair the product yourself.
2. Disconnect the device from the power supply when it is fully charged.
3. Do not use the product if any component is damaged or missing and if the device overheats or operates erratically.
4. Do not use accessories or spare parts other than those supplied.
5. Do not store the device near water or a heat source. Do not expose the device to extreme temperatures, high humidity or direct sunlight.
6. The device must be operated by an adult, do not allow children to use the product alone.
7. Do not store the device and its packaging in a place that is easily accessible to children. Keep the cable and other accessories out of the reach of children.
8. This product is not a toy. Do not allow children to play with the device.
9. Ensure that the child is breathing properly and not holding his or her breath when aspirating secretions.
10. Do not suction the child's nasal secretions while the child is sleeping.
11. To avoid irritation, do not insert the tip too deeply into the nose.
12. Test the suction power on the back of an adult's hand before use.
13. Clean the device after each use.
14. Never block the product's vents or place it on a soft surface such as a bed or sofa where the vents may be blocked.
15. Stop using the product if you notice irritation in your child's nose. Do not use the aspirator if there is nasal bleeding, blood clotting disorders or head injuries.
16. Do not use this product on your ears, mouth or eyes.

02. CONTENTS OF THE KIT

SEE FIG. A

1. Aspirator
2. Tip x3
3. Container with light
4. Tube x2 (50 and 100cm)
5. Filter x5

6. Brush
7. Flat brush
8. Tweezers
9. Type-c cable
10. Storage bag
11. Operating instructions

03. DESCRIPTION OF THE DEVICE

SEE FIG. B

1. Suction power level indicator (L0 - L9)
2. Charge level indicator
3. Suction start indicator
4. Volume indicator for pre-loaded melodies
5. Increase/decrease suction power level
6. Increase/decrease volume of pre-loaded tunes
7. Launch indicator for pre-loaded melodies
8. Melody on/off/next melody
9. Switching suction on/off
10. Switching on/off UV sterilisation
11. Switching the device on/off
12. UV sterilisation trigger indicator
13. Light illuminating the inside of the nose
14. Switching on the light for illuminating the inside of the nose
15. Tip

04. OPERATION OF THE DEVICE

1. Remove the container, tube and tip from the accessory storage drawer. Connect the tube to the aspirator and the container, then place the tip on the container. **SEE FIG. C.** Make sure the tube tips are fitted correctly.
2. Press and hold the B11 power button to start the device.
3. To start the suction, press button B9 and adjust the suction power level using adjustment buttons B5. After starting the suction, the device starts at level 1 (L1) every time. To interrupt the suction, press button B9 again.
4. To start one of the three pre-loaded melodies, press button B8. A second and third press will switch to the second and third melodies in turn. Use the B6 buttons to adjust the volume of the melodies. To switch off the pre-loaded melodies, press button B8 a fourth time.
5. To activate UV sterilisation, press button B10. UV sterilisation will automatically switch off after 20 minutes.
6. Press and hold the B11 power button to switch the device off. In standby mode, the device will automatically switch off after 10 minutes.
7. Charge the device using the cable provided. During charging, the charge level indicator illuminates. The device cannot be started while charging. Remember to disconnect the device from the power supply when it is fully charged.
8. Ensure that the child's head is in an upright position when extracting nasal secretions. Insert the tip into the nostril to seal it making sure the tip is not inserted too deeply.
9. To avoid eardrum pressure imbalance due to negative pressure, children should not hold their breath, close their mouth or swallow while using this product.

CAUTION: Do not use the device if the secretion exceeds the MAX level marked on the container. This can lead to damage to the device.

05. CLEANING AND STORAGE

1. The device should be cleaned after each use.
2. Disconnect the tube from the aspirator and the container. Then pull the tip out of the

container and slide the piece with the light upwards. **SEE FIG. D.**

3. Press the button at the bottom of the container to open it and rinse any lingering secretions under running water. Soak the container (without the light element), tube and tip in water (up to 100 °C, no more than 10 minutes) and clean with the supplied brushes and mild detergent if necessary, then leave to dry. **SEE FIG. E.**
4. The container (without the light element), tips and tubes can also be sterilised in bottle sterilisers.
5. To start UV sterilisation, insert the parts you want to sterilise into the drawer of the aspirator and then press button B10. The parts that we can sterilise with UV light are: tips, container, and tubing.
6. If you notice that the blue filter at the bottom of the container is covered with secretions, it needs to be replaced. To do this, turn the white base of the container, located at the bottom of the container, to the left and then gently pull it towards you. Replace the filter with a new one (we have an extra 5 pieces of replacement filter in the kit). **SEE FIG. F.**
7. To assemble the container after changing the filter, place the white base on the underside of the container and then turn clockwise. **SEE FIG. G.**
8. Make sure all components of the device are completely dry before reassembling or storing in the aspirator drawer.
9. Switch off the device during cleaning.
10. Do not immerse the aspirator under water. Clean it with a clean, damp cloth.
11. When not in use, the product should be stored in a dry place.

06. PROBLEM SOLVING

1. Poor or no suction:

- Check that the tip is correctly inserted into the container.
- Check that the device is charged.
- Check that other components, such as the tube, are fitted correctly.
- Check whether the filter at the bottom of the container is contaminated by secretions. If so, replace the filter.

2. Irritation and pain in the nasal cavity:

- The aspirator cannot be used if there is a nose bleed, blood clotting disorders or head injuries.
- Do not insert the tip too deeply into the nasal cavity and do not press the tip too hard against the inner walls of the nasal cavity.
- Ensure that the suction power setting is not too high.

3. Difficulty in clearing nasal secretions:

- Ensure that the baby's head is in an upright position when extracting nasal secretions.
- If necessary, spray the inside of the nasal cavity with a saline solution to dilute any lingering secretions and make suctioning easier.

07. SPECIFICATION

Input: 5V 1A

Rechargeable battery: Lithium polymer 3.7V, 1800mAh

Suction force: max 60kPa $\pm$ 5kPa

Air flow rate: 7l/min

Sterilisation: UV 5W

Container capacity: 35ml

Tip material: silicone

Sound level: $\leq$ 65dB

Charging time: approx. 2h

Operating time: up to 4h

Dimensions: 15.2x8.5x11.5cm

Weight: 510g

08. WARRANTY CARD

The product comes with a 24-month warranty. Warranty terms and conditions can be found at:

<https://neno.pl/gwarancja>

Details, contact and service address can be found at: <https://neno.pl/kontakt>

Specifications and contents are subject to change without notice. We apologise for any inconvenience.

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EN

The crossed out trash can symbol indicates that unusable electrical or electronic devices, its accessories (such as power supplies, cords) or components (for example batteries, if included) cannot be disposed of alongside with household waste. In order to dispose of the devices or its components (for example, batteries) deliver the device to the collection point, where it will be accepted free of charge. Disposal is subject to the recast version of the WEEE Directive (2012/19/ EU) and the Directive on batteries and accumulators (2006/66/ EC). Proper disposal of the device prevents degradation of the natural environment. Information about the collection points of the facilities is issued by the competent local authorities. Incorrect disposal of waste is subject to penalties provided for by the law in force in the given area.

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