nenor[®]

Sole Twin

Table of contents



Spis treści

7	Instrukcja obsługi	PL
13	User manual	EN
20	Bedienungsanleitung	DE
27	Návod k použití	CZ
33	Használati utasítás	HU
40	Používateľská príručka	SK
46	Användarhandbok	SE
53	Käyttäjän käsikirja	FI
59	Brugerhåndbok	NO
65	Brugermanual	DK
72	Gebruikershandleiding	NL
78	Manual del usuario	ES
85	Manuale utente	IT
92	Manuel de l'utilisateur	FR
98	Manual de utilizare	RO
105	Korisnički priručnik	HR/BA
111	Uputstvo za upotrebu	RS/ME
117	Uporabniški pravilnik	SI
124	Εγχειρίδιο χρήστη	GR
131	Lietotāja rokasgrāmata	LV
137	Vartotojo vadovas	LT
143	Kasutusjuhend	ET
150	Посібник користувача	UA
157	Ръководство за потребителя	BG
164	Упатство за употреба	MK
171	Manual do utilizador	PT


A

B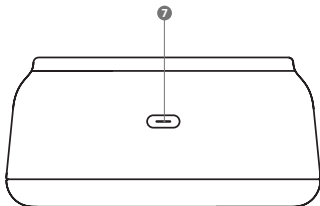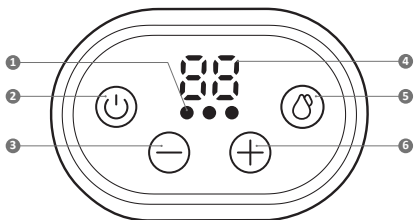

C

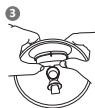D

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USER MANUAL

Dear Customer,

The Neno Sole Twin device you have purchased is an dual, electronic, three-phase breast pump used to express breast milk from a woman's breast after giving birth. Read the instruction manual before use.

01. PRECAUTIONS

1. Always unplug the device after charging if you do not need to charge the device's battery.
2. Do not place the product where it is in danger of falling into sinks, bathtubs or other containers.
3. The breast pump and its components are not heat resistant – keep the device away from hot surfaces and open flames.
4. Do not use the device near flammable materials.

5. The breast pump should not be left unattended when connected to a power source.
6. Never use the device if it has been dropped or otherwise damaged.
7. Do not use a device whose power cord or plug is damaged.
8. Do not use the device in the bath or shower.
9. Do not place the device under water or in other liquids.
10. Do not wash the unit under running water.
11. Do not connect the device to a power socket that has been in contact with water or other liquids.
12. If the device has been exposed to water or other liquids - do not touch the device directly, disconnect it from the power supply, switch it off and contact the manufacturer.
13. This appliance is intended for use by a single user. Use of the breast pump by more than one person carries a risk of harm to the users and voids the warranty.
14. Do not repair or modify the unit yourself.
15. Use the breast pump only as described in the instructions. Do not use the device for any other purpose.
16. Inspect all breast pump components before each use.
17. Do not heat the discharged milk in the microwave. Milk heated in a microwave can burn the baby's face because heating with microwave waves can create high temperature bubbles in the liquid. In addition, microwave waves can change the composition of the milk.
18. Do not bring the milk to the boil while heating it.
19. Before using the breast pump for the first time, wash and disinfect all parts of the machine that will come into direct contact with your breast or milk.
20. After each use of the breast pump, wash all parts of the machine that come into direct contact with your breast or milk.
21. Take special care when using the breast pump or its accessories near children.
22. Do not use the breast pump while charging.

02. BEFORE FIRST USE

1. Connect the breast pump to the power supply. Charge the batteries for 2,5 hours.
2. Remove the breast pump.
3. Disconnect the parts of the breast pump that will come into direct contact with the breast or milk: remove the breast shield from the milk container, then pull the valve down and the membrane up to disconnect from the connector.

4. Clean the appliance components (refer to the **CLEANING AND DISINFECTION** section).

NOTE!

Use the breast pump in an upright position. Use in any other position (e.g. lying down) will result in the breast pump not fitting properly to the breast and milk leaking out.

03. ELEMENTS

FIG. A

1. Pumping device x2
2. Milk container x2
3. Connector x2
4. Valve x4
5. USB-C charging cable x2
6. Breast shield 24mm x2
7. Membrane x4
8. Breast shield reducer 20mm x2

04. DESCRIPTION OF THE PUMPING DEVICE

FIG. B

1. Icon of the selected mode (I dot - stimulation phase, II dots - deep extraction phase, III dots - mixed phase).
2. Power button - Hold the button for 2 seconds to switch the device on or off. When the device is on press to pause the breast pump.
3. Changing the suction level - Press “-” to reduce the suction power.
4. Icon of the selected mode (L1-L8)/time of the breast pump.
5. Mode change button - Press the button to change the operating mode of the unit.
6. Changing the suction level - Press “+” to increase the suction power.
7. USB-C charging socket

05. CLEANING AND DISINFECTION

Washing and disinfecting are two different operations. They must be carried out separately to protect your baby and to ensure that the breast pump works properly.

Washing - washing the surface of the unit and removing dirt.

Disinfection - killing organisms such as bacteria or viruses that may be present on the surface of the equipment.

Breast shield, membrane, valve, reducer, connector, milk container:

Wash and disinfect - before first use.

Wash - after each use.

Once a day and before first use - disinfection.

A. WASHING:

- Disassemble all parts of the breast pump (except the pumping device) and wash in warm soapy water for 5 minutes, then rinse. Do this immediately after using the machine. This will remove milk deposits and prevent bacterial growth.
- Only use drinkable tap water for washing. Use bottled water if the water in your tap does not meet this requirement.
- When cleaning silicone parts, do not use brushes. Be careful not to pierce them through.
- Use a mild dishwashing liquid or soap.
- Allow the parts to dry naturally. Store dry appliance parts in a cool, dry place when you are not using them.

B. DISINFECTION:

Membrane, valve, reducer, connector, breast shield, milk container:

- Fill the pot or steriliser with water so that all the components inside (membrane, valve, reducer, connector, breast shield, milk container) are covered when placed inside. Bring to the boil. Place breast pump parts in boiling water for 10 minutes. Do not boil.
- Allow the pieces to dry naturally. Store dry kit items in a cool, dry place when you are not using them.

C. CLEANING THE PUMPING DEVICE:

- Wipe the pump unit with a soft, wet cloth and a mild dishwashing liquid/soap. Other cleaners or abrasives may damage the unit and its display.
- When not using the pump unit, store it in a cool, dry place.

06. ASSEMBLY AND USE OF THE BREAST PUMP

FIG. C

Attach the components as shown in FIGS. C 1-5.

NOTE: Wash your hands thoroughly with soap and water before touching the breast pump and breast.

1. Insert the membrane into the connector. **FIG. C1.**
2. Connect the valve to the connector. **FIG. C2. ATTENTION:** The valve should be fixed with its longer part towards the breast shield. For an overview: **FIG. C6.**
3. Attach the connector with the membrane and valve to the milk container, starting by placing the membrane in the groove. **FIG. C3.**
4. Place the breast shield in the milk container. **FIG. C4.**
5. Attach the pumping device to the top of the milk container. **FIG. C5.**
6. Place the breast shield on your breast so that the nipple is inside the

drainage channel. Press the breast shield and breast pump connector against your breast using your thumb and index finger. **FIG. C6.**

7. Insert the breast pump into the bra/lactation corset. **FIG. C7.**
8. Turn on the breast pump by pressing the power button and select the operation mode that suits you.
9. To pour the milk hold the breast pump horizontally and carefully unscrew the pump **FIG C8.** Hold the milk container with the breast shield facing upwards - lay it flat on the table top (do not lay it upwards to prevent the milk from flowing out). **FIG C9.** Carefully pour the milk into the bottle/storage container. **FIG C10.**
10. Remove and clean all breast pump components after use. **FIG C11.**

NOTES:

- a. Before each use, check the parts of the breast pump for damage. If necessary, replace damaged parts with new ones.
- b. Always check the cleanliness of components before using the unit.
- c. If the diameter of the breast shield is too large, use the supplied reducer by inserting it inside. **FIG. C12.**

NOTES:

The breast pump starts in stimulation mode. Set the level of suction strength comfortable for you using the +/- buttons:

- Stimulation mode - fast pumping that mimics your baby's natural sucking of breast milk to stimulate milk flow. After 2 minutes, the breast pump will automatically activate the deep pumping mode. Adjust the suction level using the +/- buttons to find the highest level of suction that is comfortable for you. If milk starts to flow from your breast before two minutes have elapsed, press the mode change button to manually switch to deep extraction mode.
- Deep pumping mode - slower pumping that gently and efficiently sucks milk from the breast. In this mode increase the suction until you feel slight discomfort (not pain) and reduce the suction by one degree.
- Mixed mode - mimics the baby's sucking reflex. Can be used after pumping to help empty the breast and increase lactation.

07. CHARGING THE BREAST PUMP

Fully charge the breast pump battery before putting it into storage if you intend not to use the device for a long period of time. Charge the device battery before it is completely exhausted. This extends the life of the battery. If the breast pump has been stored in a hot place, the device battery may not work at first. Allow the breast pump to cool down for one hour before you start using it.

Indications:

LO - displays if the battery level is below 20%. **FIG D1.**

Flashing dots - breast pump under charging. **FIG D2.**

Three dots glowing continuously- battery fully charged. **FIG D3.**

Do not use the breast pump while charging.

08. REPLACEMENT OF CONSUMABLES

Consumable components are subject to natural wear and tear during operation of the breast pump and are therefore not covered by the manufacturer's warranty.

Worn components of the breast pump that come into contact with milk should be replaced. A worn component has a crack, deformation, tarnishing or a distinct change in colour.

- The membrane should be replaced every 8 weeks (if you use the breast pump 1-3 times a day) or every 3-4 weeks (if you use the breast pump more than 3 times a day).
- The valve should be replaced every 2-3 months (if you use the breast pump 1-3 times a day) or every 4 weeks (if you use the breast pump more than 3 times a day).
- The breast shield and connector should be replaced every 6 months if you use the breast pump more than 3 times a day.

09. LONG-TERM STORAGE

The breast pump has a built-in rechargeable battery. To protect itself from damage, the device enters storage mode after a long period of inactivity. To deactivate the storage mode, connect the device to power and charge it for 2 hours. Before you put the breast pump away for a long period of time, make sure that:

- All components of the breast pump are clean.
- You have cleaned the pumping device.
- The device's battery is fully charged.

10. FAQ

1. **The device does not start:** make sure the breast pump is plugged into a working electrical outlet. If the breast pump has no power source, its display will not turn on and no error message appears.
2. **No/poor suction:** make sure all breast pump parts are positioned correctly. Take the breast pump apart and make sure all parts are clean and undamaged. Reassemble the breast pump according to the instructions in the LACTATOR ASSEMBLY section. When pumping, make sure the

funnel is fully seated against the breast.

- 3. More milk has flowed from the breast than the container can hold:**
use a damp (not wet) cloth to wipe the breast pump. Wash the parts of the breast pump that hold milk according to the instructions in the CLEANING AND DISINFECTING section. When all parts of the machine are dry, reassemble.

11. SPECIFICATION

Capacity of milk container: 210ml

Battery capacity: 3.7V 1500 mAh Li-Po

Input power: 110-240V

Output power: 5V 1A

Operating time: up to 150min

Charging time: approx. 150min

Dimensions of breast pump: 9x4,5x6 cm x2

Weight of breast pump: 150g x2

Suction power:

1. Stimulation mode (8 levels): 9-24 kPa
2. Deep extraction mode (8 levels): 13-40 kPa
3. Mixed mode (8 levels): 10-24 kPa

Permissible variation ± 5 Kpa.

12. WARRANTY/ COMPLAINTS CARD

The product comes with a 24-month warranty. Warranty conditions can be found at: <https://nenopl/gwarancja>

Details, contact and service address can be found at:

<https://nenopl/kontakt>

Specifications and contents are subject to change without notice. We apologise for any inconvenience.

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EN

The crossed out trash can symbol indicates that unusable electrical or electronic devices, its accessories (such as power supplies, cords) or components (for example batteries, if included) cannot be disposed of alongside with household waste. In order to dispose of the devices or its components (for example, batteries) deliver the device to the collection point, where it will be accepted free of charge. Disposal is subject to the recast version of the WEEE Directive (2012/19/ EU) and the Directive on batteries and accumulators (2006/66 / EC). Proper disposal of the device prevents degradation of the natural environment. Information about the collection points of the facilities is issued by the competent local authorities. Incorrect disposal of waste is subject to penalties provided for by the law in force in the given area.

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