**nenò**<sup>®</sup>

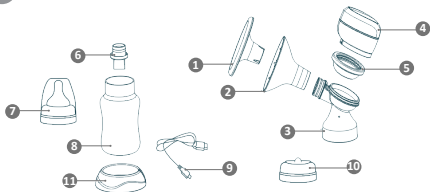
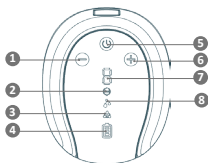
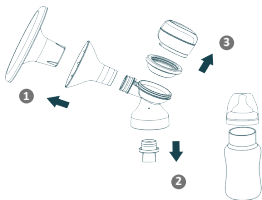
Angelo

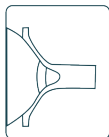
# Table of contents

## Spis treści

- 5** Instrukcja obsługi
- 10** User manual
- 15** Bedienungsanleitung
- 20** Manuel de l'utilisateur
- 26** Návod k použití
- 31** Používateľská príručka
- 36** Használati utasítás
- 41** Användarhandbok
- 46** Käyttäjän käsikirja
- 51** Brukerhåndbok
- 56** Brugermanual
- 61** Gebruikershandleiding
- 66** Manual del usuario
- 72** Manuale utente
- 77** Manual de utilizare

**A****B****C**

**D****E**

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## **USER MANUAL**

Dear Customer,

Thank you for buying the electronic breast pump Neno Angelo. Please refer to the user manual before use.

### **01. SAFETY**

1. Do not use the breast pump during pregnancy – it can cause preterm birth.
2. Do not use the pump if you suffer from chronic diseases or hypersensitivity, which could make it painful for you to use.
3. Before each use make sure that no part of the breast pump is damaged. Do not use the device if any of its elements are visibly damaged.
4. Keep the pump out of reach of children and animals.
5. Due to hygienic reasons, only one person should use breast pump.
6. Keep the pump elements away from running water.
7. Avoid exposing the device to mechanical damage caused by, for example, dropping it.
8. Keep the silicone parts away from hot surfaces, to avoid deformation of the material.
9. Before storing the expressed milk make sure to clean and sterilize the breast pump each time.
10. Do not clean the device with abrasive cleaners.
11. Do not try to repair the breast pump yourself, it will cause loss of warranty.
12. If the breast pump causes you pain and discomfort during use, turn it off and move it away from your breast.
13. Do not plug the device into a voltage higher than stated in the specification.

## 02. CHARGING

Charge the breast pump 2 hours before first use. To charge, connect the pump to the output with the USB cable. The battery icon on the display will appear full when charging is complete.

## 03. DESCRIPTION OF MODES

### Stimulation mode:

L1 – 6 kPa; L2 – 13 kPa; L3 – 18 kPa; L4 – 22 kPa ; L5 – 24 kPa ;L6 – 29 kPa; L7 – 31 kPa; L8 – 34 kPa ; L9 – 36 kPa ;

### Expression mode:

L1 – 22 kPa; L2 – 26 kPa; L3 – 28 kPa; L4 – 31 kPa ; L5 – 33 kPa ; L6 – 38 kPa ; L7 – 40 kPa; L8 – 43 kPa; L9 – 46 kPa;

### Let-down mode:

L1 fast - 14 kPa; L1 slow - 22 kPa ; L2 fast – 14 kPa ; L2 slow – 24 kPa ;L3 fast – 14 kPa; L3 slow- 28 kPa ; L4 fast – 14 kPa; L4 slow – 30 kPa; L5 fast – 14 kPa; L5 slow – 33 kPa; L6 fast – 14 kPa; L6 slow – 38 kPa; L7 fast – 14 kPa; L7 slow- 40 kPa ; L8 fast – 14 kPa; L8 slow – 43 kPa ;L9 fast – 14 kPa ; L9 slow – 46 kPa ;

Permissible difference  $\pm 5$  Kpa.

## 04. ELEMENTS

SEE FIG. A

1. Cap
2. Breast shield
3. Connector
4. Pumping device
5. Suction Bowl
6. Valve
7. Milk bottle
8. Set with bottle teat
9. Charging Cable
10. Bottle lid

## 05. DESCRIPTION OF ICONS AND BUTTONS

### 1. Power button:

Press and hold the button for 2 seconds to turn the device on or off.

### 2. Mode change:

To change modes press the button **SEE FIG. E**. The device will automatically switch from stimulation to expression mode after 2 minutes. Additionally, the breast pump saves previous settings and is equipped with a timer. After turning on the device, the last used settings will appear. The device will automatically turn off after 30 minutes of use.

### 3. Suction power:

To increase the suction power, press the + button on the panel; to decrease the suction power, press the – button on the panel (Stimulation mode: L1-L9;

Expression mode: L1-L9; let-down mode L1-L9)

**4. Battery status:**

Indicates the remaining battery power. Charge the breast pump if the battery is low.

**5. Timer:**

Indicates the working time of the pump. The device will automatically turn off after 30 minutes.

Note: Press and hold the + button to turn off the backlight

**SEE FIG. B**

1. Suction decrease
2. Stimulation mode
3. Let-down mode
4. Battery percent
5. Power button/ Mode change
6. Suction increase
7. Suction level
8. Expression mode

**06. TROUBLESHOOTING**

1. The device does not start: make sure that the breast pump is connected to a working power outlet. If the device is not connected, the display panel will not light up and an error message will not appear on the screen.
2. Poor/ no suction: make sure that all the elements of the pump are correctly positioned. Disassemble the device and make sure that all of the parts are clean and not damaged. Assemble the device according to the instructions in the “assembling the breast pump” section.
3. The breast produced more milk than the bottle can hold: Turn the device off and unplug it from the outlet. Use a damp (not wet) cloth to wipe the breast pump. When all elements of the device are dry, reassemble them.

**07. BEFORE FIRST USE**

**SEE FIG. C**

1. Remove the cap from the breast shield each use.
2. Pull the valve down to remove it from the connector.
3. Separate the connector from the pump.

**08. CLEANING**

Cleaning and sanitizing are two different things. They need to be done separately, to protect your child and to ensure proper functioning of the breast pump.

**Cleaning** – cleaning the surface and getting rid of stains and dirt.

**Sanitizing** – getting rid of the bacteria and viruses that can be present on the surface of the device.

	Breast shield	Breast pump elements (connector, suction cap, suction bowl)	The bottle and teat
Before first use	Cleaning and sanitizing	Cleaning and sanitizing	Cleaning and sanitizing
After each use	Cleaning	Cleaning	Cleaning
Once a day	Sanitizing	Sanitizing	Sanitizing

### 09. REPLACEMENT OF CONSUMABLE PARTS

Consumable parts are subject to natural wear and tear during use of the breast-pump and are therefore not covered by the manufacturer's warranty.

Worn parts of the breastpump that come into contact with milk should be replaced.

A worn part has a crack, is deformed, or you notice discolouration or a marked change in colour.

- Replace the membrane every 8 weeks (if you use the breastpump 1-3 times a day) or every 3-4 weeks (if you use the breastpump more than 3 times a day).
- Replace the valve every 2-3 months (if you use the breastpump 1-3 times a day) or every 4 weeks (if you use it more than 3 times a day).
- The breastshield should be replaced every 6 months if you use the breastpump more than 3 times a day.
- Replace the tube only if moisture gets into the tube or if it is damaged.

### 10. ASSEMBLING THE BREAST PUMP

- Wash your hands thoroughly with water and soap before touching the breast and the breast pump.
- Connect the breast shield, suction bowl and suction cap to the connector.
- Connect the breast shield to the connector.
- Connect the valve to the connector.
- Place the valve in the connector and screw the milk bottle onto the connector by turning it clockwise until it sits tightly in place.

Tips:

- Before each use, check the parts of the breast pump for damage. If any of the parts appear to be damaged, replace them with new ones.
- Before each use, check if all the parts of the device are clean.
- To avoid damaging the breast pump, make sure that all its elements are dry before use.

### 11. LONG-TERM STORAGE

- The breast pump has a built-in battery accumulator. To save the accumulator from damage, the device switches into a storage mode after a long period of

inactivity. To turn the storage mode off, plug the device into a power outlet and charge it for 2 hours.

2. Before you put the breast pump away for a long time, make sure that:
  - a. All components of the breast pump are clean.
  - b. The pumping device is clean.
  - c. The battery is fully charged.

## 12. USING THE BREAST PUMP

### SEE FIG. D

1. Place the breast shield on the breast so that the nipple is inside the drain canal.
2. Use your thumb and index finger to press the breast shield onto your breast. Support your breast with your hand.
3. Turn on the breast pump by pressing the power button and select a mode on the display.
4. After use, press the power button to turn off the breast pump and place a lid on the breast shield to keep it from accumulating dust.

- a. The breast pump starts work in the stimulation mode. Set a comfortable suction level for you, using the +/- buttons

**Stimulation mode** – fast pumping that mimics the natural sucking motion of a child and stimulates the milk flow.

- b. After 2 minutes, the breast pump will automatically switch to the expression mode. Set the highest most comfortable suction level for you using the +/- buttons. If the milk begins to flow out of your breast before 2 minutes pass, press the change mode button to switch to manual milk expression.

**Expression mode** – slower pump motions that gently suck the milk out of the breast.

- c. When using the expression mode, increase the sucking power until you feel a slight discomfort (not pain) and lower the suction power by one level.

**The let down** – mode mimics the same sucking pattern of baby by programming the suction speed and strength of suction. It can be used after expression mode to help mothers to empty the breast and increase your milk supply.

## 13. SPECIFICATION

**Input power:** 110-240V 50-60Hz

**Output power:** 5V 1A

**Battery:** 3.7 1400mAh Li-ion

**Pump size:** 82x67x64mm

**Weight:** 392g

## 14. WARRANTY CARD

The product is covered by a 24-month warranty.

The terms of the guarantee can be found at: <https://nenop.pl/warranty>

Details, contact and website address can be found at: <https://nenop.pl/kontakt>

Specifications and contents of the kit are subject to change without notice. We apologise for any inconvenience.

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EN

The crossed out trash can symbol indicates that unusable electrical or electronic devices, its accessories (such as power supplies, cords) or components (for example batteries, if included) cannot be disposed of alongside with household waste. In order to dispose of the devices or its components (for example, batteries) deliver the device to the collection point, where it will be accepted free of charge. Disposal is subject to the recast version of the WEEE Directive (2012/19/ EU) and the Directive on batteries and accumulators (2006/66 / EC). Proper disposal of the device prevents degradation of the natural environment. Information about the collection points of the facilities is issued by the competent local authorities. Incorrect disposal of waste is subject to penalties provided for by the law in force in the given area.

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