**nenò**<sup>®</sup>

Claro

## Table of contents

### Spis treści

<b>3</b>	Instrukcja obsługi	<b>PL</b>
<b>4</b>	User manual	<b>EN</b>
<b>6</b>	Bedienungsanleitung	<b>DE</b>
<b>8</b>	Návod k použití	<b>CZ</b>
<b>10</b>	Használati utasítás	<b>HU</b>
<b>11</b>	Používateľská príručka	<b>SK</b>
<b>13</b>	Användarhandbok	<b>SE</b>
<b>15</b>	Käyttäjän käsikirja	<b>FI</b>
<b>16</b>	Brukerhåndbok	<b>NO</b>
<b>18</b>	Brugermanual	<b>DK</b>
<b>20</b>	Gebruikershandleiding	<b>NL</b>
<b>21</b>	Manual del usuario	<b>ES</b>
<b>23</b>	Manuale utente	<b>IT</b>
<b>25</b>	Manuel de l'utilisateur	<b>FR</b>
<b>27</b>	Manual de utilizare	<b>RO</b>

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## USER MANUAL

Dear Client,

The NENO CLARO you have purchased is a milk collector for collecting milk during feeding / pumping from the other breast. It aims to collect food in a simple and effective way. The kit includes two milk collectors with a storage case.

### 01. CAUTIONS

1. Before each use, please check the condition of the product.
2. We recommend using only soft brushes or sponges for cleaning. Hard materials can scratch the product.
3. If there is any sign of damage, replace the product.
4. Do not store near sharp objects.
5. This product is not a toy. The collector should be used in accordance with its intended use.
6. Store the product in a dry place and avoid direct sunlight.

## 02. PRODUCT FEATURES

### Easy to use

Just put the product on the breast — the collector starts sticking and allows milk to be collected.

### BPA free

Made of high-quality silicone and PP. BPA free.

### Painless, quiet operation

The collector allows for discreet use. It can be easily hidden in a bra or corset.

### Get rid of leaks

When you are breastfeeding with one breast, milk flows from the other breast by itself. It is a perfect device to avoid staining your underwear and at the same time not to waste precious food. The collector gathers every drop of milk flowing out of the breast that you are not feeding your baby at the same moment.

### Easy to clean

For sterilization, prepare a vessel with boiling water and place the disassembled parts of the product in it. Sterilization should take approximately 3-4 minutes.

## 03. HOW TO USE THE COLLECTOR

1. Make sure that all the components are properly installed.
2. Hold the product on your breast corking the slot.
3. The collector should be used while wearing a bra. Make sure it is worn properly to prevent leakage.
4. When the container is full, remove the silicone cap and pour the milk into a feeding bottle or pour it into a milk bag and refrigerate.
5. Wash the collector under running water.
6. Pass one end of the silicone pin through the hole on the other end to hang the collector to dry.
7. Place in a cool and well-ventilated place to dry.

## 04. SPECIFICATION

**Material:** Silicone (overlay) + PP (container)

**Case material:** PP

**Collector dimensions:** 10.3 x 9.5 x 3.5 cm

**Set dimensions:** 11x7cm

**Collector weight:** 32g

**Set weight:** 114g

**Set:** 2 pcs of collector, case

#### **05. WARRANTY CARD**

The product is covered by a 24-month warranty. The terms of the guarantee can be found at: <https://nenopl/gwarancja>

Details, contact and website address can be found at: <https://nenopl/kontakt>

*Specifications and contents of the kit are subject to change without notice. We apologise for any inconvenience.*

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