**nenob**<sup>®</sup>

Leite





## Table of contents

### Spis treści

<b>4</b>	Instrukcja obsługi	<b>PL</b>
<b>5</b>	User manual	<b>EN</b>
<b>7</b>	Bedienungsanleitung	<b>DE</b>
<b>9</b>	Návod k použití	<b>CZ</b>
<b>10</b>	Használati utasítás	<b>HU</b>
<b>12</b>	Používateľská príručka	<b>SK</b>
<b>14</b>	Användarhandbok	<b>SE</b>
<b>15</b>	Käyttäjän käsikirja	<b>FI</b>
<b>17</b>	Brukerhåndbok	<b>NO</b>
<b>19</b>	Brugermanual	<b>DK</b>
<b>20</b>	Gebruikershandleiding	<b>NL</b>
<b>22</b>	Manual del usuario	<b>ES</b>
<b>24</b>	Manuale utente	<b>IT</b>
<b>25</b>	Manuel de l'utilisateur	<b>FR</b>
<b>27</b>	Manual de utilizare	<b>RO</b>

A

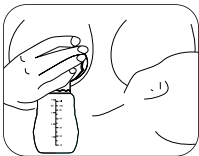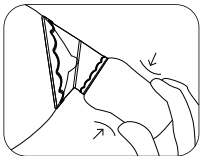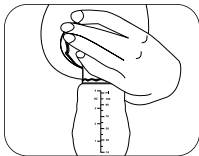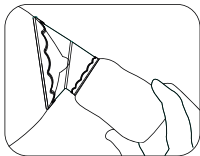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# EN

## USER MANUAL

Dear Customer,  
thank you for purchasing Neno Leite, a silicone milk collector for collecting milk during feeding/pumping from the other breast. It aims to collect food in a simple and effective way.

### 01. SAFETY MEASURES

1. Before each use, check the condition of the product.

2. We recommend using only soft brushes or sponges for cleaning, hard materials may scratch.
3. If there is any sign of damage, replace the product.
4. Do not store near sharp objects.
5. This product is not a toy. The collector should be used in accordance with its intended use.
6. Store the product in a dry place and avoid direct sunlight.

## **02. PRODUCT FEATURES**

### **Easy to use**

With just one squeeze, the milk collector draws out your milk with the power of continuous suction-no noise, no electricity and no cords.

### **BPA FREE Material**

Made of high quality of silicone. No BPA, PCV and phthalate.

### **Painless and work silent**

The milk collector helps to express breast milk discreetly, silently and quickly. Avoid embarrassment in public places.

### **No Leak Any More**

When most mother are breastfeeding, the breast on the other side would often spill breast milk which may wet your clothes or your baby, or be wasted on the disposable nursing pad. The milk collector can collect every drop of breast milk while you're breastfeeding.

### **Easy to clean**

All-in-one design and no disassemble required. They are very easy to clean by clear boiling water for about 2 minutes.

## **03. HOW TO USE**

### **SEE FIG. A**

1. Make sure your milk collector has been cleaned before and after use.
2. Squeeze the pump to push internal air out.
3. Invert the flange, place it onto your nipple centered in the middle of the breast shield.
4. Restore the flange, then the pump would firmly suck onto your breast.
5. You may use the product as a collector. The milk will then flow freely to the pump. If you want to use it as a manual breast pump - squeeze the pump to facilitate milk leak.

#### 04. SPECIFICATION

**Material:** Silicone+PP

**Capacity:** 100 ml

#### 05. WARRANTY CARD

The product is covered by a 24-month warranty.

The terms of the guarantee can be found at: <https://nenopl/gwarancja>

Details, contact and website address can be found at: <https://nenopl/kontakt>

*Specifications and contents of the kit are subject to change without notice.*

*We apologise for any inconvenience.*

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