nenò[®]

Mio

Table of contents

Spis treści

3	Instrukcja obsługi	PL
5	User manual	EN
8	Bedienungsanleitung	DE
10	Návod k použití	CZ
13	Használati utasítás	HU
15	Používateľská príručka	SK
18	Användarhandbok	SE
20	Käyttäjän käsikirja	FI
23	Brukerhåndbok	NO
25	Brugermanual	DK
27	Gebruikershandleiding	NL
30	Manual del usuario	ES
33	Manuale utente	IT
35	Manuel de l'utilisateur	FR
38	Manual de utilizare	RO

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USER MANUAL

Dear Customer,

The Neno Mio product you have purchased is a baby bottle with a teat that can be used for feeding and drinking, as well as for expressing breastmilk using all Neno brand breast pumps. Before using the bottle, it is necessary to read the accompanying instructions. The bottle is suitable for babies aged 6 months and over.

01. PRECAUTIONS

1. Do not allow your child to use this product without adult supervision.
2. Do not allow your child to play with the parts of the bottle or run with it.
3. Never use a bottle teat as a normal teat.
4. Always check the temperature of the contents of the bottle before feeding your baby.
5. Keep all components out of the reach of children.

6. Wash the product before first use. Check the condition of the product carefully before each use and stretch the teat in all directions with your fingers. If the teat shows any signs of damage or wear, discard it. Do not leave the teat in direct sunlight, in a hot place or in a steriliser for longer than recommended as this may damage the product.
7. Do not place the product in a hot oven.
8. Place the teat in boiling water for 5 minutes before first use. Clean before each use.
9. Do not place the bottle in a microwave oven. Do not use microwave waves to heat the contents in the bottle.
10. Do not use abrasive or antibacterial agents when cleaning the product. Do not place product parts directly on surfaces that have been cleaned with antibacterial agents.
11. Too much concentration of cleaning agents can cause the plastic to crack. In case of visible cracks in the product, discard it immediately.
12. Sterilisation and high temperatures can lead to distortion and the cap not fitting into the bottle.
13. The bottle is dishwasher safe in the top basket.
14. Boiling the bottle and teat in hard water can cause sediment to form - a natural process resulting from the hardness of the water.

02. CONTENTS OF THE KIT

1. Bottle
2. Teat size +6m
3. Cap
4. Lid
5. Silicone weighted straw
6. Handle
7. Needle
8. Brush

Sucking is an important part of a baby's development, so when feeding a bottle with a teat, make sure that the child can suck actively and effortlessly.

03. MONTAGE

Place the lid on the bottle so that the teat is in an upright position. To remove the lid, place your hand on it by placing your thumb against the bottom of the lid. The decorative element on the lid is easy to remove for hygienic reasons. This keeps the teat area hygienic and the nooks easy to clean, so do not pull on this element when removing the lid. It is easier to insert the teat if you move it sideways rather than pulling it through the hole in a straight line. Pull the teat through the hole until the base is level with the retaining ring. Attach a white weight to the tube. Fit a connector to the other end. Fit the whole assembly into the teat holder. The needle

(shaped like a teddy bear) supplied is for cleaning the air outlet of the teat.

04. CLEANING

Before using the bottle for the first time, take it apart, clean it thoroughly and sterilise it using a bottle steriliser or boiling water. The sterilisation process should take no more than 10 minutes. After each use, disassemble the product and wash the parts using warm water and detergent, then rinse. Before using sterilised items, carefully wash your hands and the surfaces on which the items will be placed. To clean the weighted straw, use a straw brush that fits its width. Insert the brush into the straw and pull through several times to clean properly. If the weight bore is clogged, use a plastic needle to unclog the flow.

05. CONSERVATION

Check the condition of the product carefully before each use and stretch the teat in all directions with your fingers. Throw away the teat when you notice signs of damage or wear. Do not leave the teat in direct sunlight or in hot places, or soak it in a sterilising agent for longer than recommended, as this may damage the product. For reasons of hygiene, it is recommended that teats be replaced every 3 months. Store teats in a dry, closed container. Do not use the bottle if there are any cracks on it. Do not allow children to use the bottle without adult supervision.

06. SPECIFICATION

1. Anti-colic valve
2. Material of construction: PPSU, PP, Silicone
3. BPA-free
4. Convenient scale on the bottle
5. Dimensions: $\varnothing 7.5 \times 12.5 \times 22 \text{ cm}$
6. Weight: 182g
7. Bottle capacity: 300 ml

07. GUARANTEE CARD

The product comes with a 24-month warranty. Warranty terms and conditions can be found at: <https://nenop.pl/gwarancja>

Details, contact and service address can be found at: <https://nenop.pl/kontakt>
Specifications and contents are subject to change without notice. We apologise for any inconvenience.

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