nenò[®]

Pesca

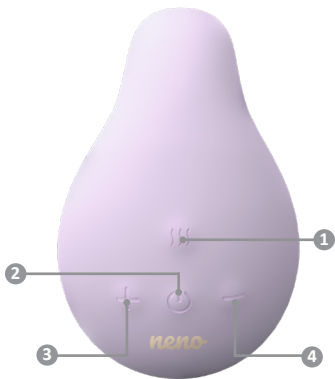
Table of contents

Spis treści

4	Instrukcja obsługi	PL
6	User manual	EN
8	Bedienungsanleitung	DE
11	návod k použití	CZ
13	Használati utasítás	HU
16	Používateľská príručka	SK
18	Användarhandbok	SE
20	Käyttäjän käsikirja	FI
22	Brukerhåndbok	NO
25	Brugermanual	DK
27	Gebruikershandleiding	NL
30	Manual del usuario	ES
32	Manuale utente	IT
35	Manuel de l'utilisateur	FR
37	Manual de utilizare	RO

A

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USER MANUAL

Dear Customer,

Thank you for choosing the Neno Pesca. The device you have purchased is a lactation massager with a heating function, which is ideal for the prevention of breast inflammation, provides relief from lactation stasis, reduces breast pain and swelling and can increase milk production. The device can be applied to and around the breast.

01. PRECAUTIONS

Fully charge the massager before using it for the first time. Never use the device while it is charging.

Use only the supplied accessories. Use of any other charger may cause damage to the device, short circuit, electric shock or fire.

Keep out of the reach of children. This is not a toy.

Under no circumstances use the device if it is damaged. Contact an authorised service centre.

Make sure the massager and your hands are dry before plugging in to charge, and keep the device away from water while charging.

Never fully submerge the device under water.

02. DESCRIPTION OF THE BUTTONS

SEE FIGURE A

1. Heating function button
2. Power button
3. Button to increase vibration intensity
4. Button to reduce vibration intensity

03. USE OF THE DEVICE

Switch on the massager by pressing and holding the power button for 2 seconds. To change the type of vibration press the power button selecting from 5 levels in sequence. Reduce or increase the intensity of the vibration using the „+“ and „-“ buttons.

To turn off the vibration press and hold the power button.

Press and hold the „heating function“ button for 2 seconds to switch the heating function on or off. During heating, the heating function button will be illuminated in pink, depending on the ambient temperature this may take approximately 6 minutes. You can also use the appliance while it is heating up. Once the appliance has heated up to level 1, the button will illuminate blue. Once the unit has heated up to level 1, we can select the 2nd heating level. To select level 2, press the heating function button and it will illuminate pink again. Once heated to level 2, the button will illuminate red. To change the heating intensity press the button again selecting one of the 2 levels.

The device allows you to use both functions independently, so you can use only vibration or heating, or both options at the same time.

To charge your device, connect the power cable to the USB port on your computer, adapter or other power source. Make sure the cable is properly connected to the device. When properly connected for charging, the power button will blink blue to indicate that the device is charging. When the device has finished charging the blue light will stop blinking.

04. SPECIFICATION

Material: silicone

Battery: Li-ion 500 mAh

Charging time: approx. 1.5 hours

Heating and vibration time simultaneously: up to 60 minutes

Dimensions: 100x65x36 mm

Weight: 117 g, 2 heating levels

Vibration: 5 levels of vibration and 3 levels of vibration intensity

Heating function: 2 levels: 42°C and 45°C

Input: 5V 1A

Charging: magnetic charger

05. VIBRATION SPECIFICATIONS

Data for vibration levels for the first intensity level:

Mode 1- operating period: 32ms; duty cycle: 90%; frequency: 31Hz

Mode 2- work period: 112ms; duty cycle: 70%; frequency: 8.9Hz

Mode 3 operating period: 10ms; duty cycle 50%; frequency: 102.65Hz

Mode 4-period: 370 ms, duty cycle: 41%; frequency: 2.7Hz

Mode 5- duty period: 188 ms 3 cycles, 46ms 3 cycles; duty cycle: 44%; frequency: 2.3Hz

06. CONTENTS OF THE KIT

Lactation massager, magnetic charger, velvet pouch, instruction manual

07. CLEANING AND STORAGE

It is recommended to clean with a soft, water-moistened cloth before and after each use. Make sure the device is switched off and not plugged in to charge.

Store the massager in a dry and cool place.

08. WARRANTY CARD

Dear Customer, thank you for purchasing our Neno Pesca massager. If you have any problems with the operation of the device under normal conditions, please contact an authorised Neno brand service centre or distributor. Keep your warranty card in case of repair.

The product comes with a 24-month warranty. Warranty conditions can be found at: <https://neno.pl/gwarancja>

Details, contact and service address can be found at: <https://neno.pl/kontakt>

Specifications and contents are subject to change without notice. We apologise for any inconvenience.

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EN

The crossed out trash can symbol indicates that unusable electrical or electronic devices, its accessories (such as power supplies, cords) or components (for example batteries, if included) cannot be disposed of alongside with household waste. In order to dispose of the devices or its components (for example, batteries) deliver the device to the collection point, where it will be accepted free of charge. Disposal is subject to the recast version of the WEEE Directive (2012/19/ EU) and the Directive on batteries and accumulators (2006/66 / EC). Proper disposal of the device prevents degradation of the natural environment. Information about the collection points of the facilities is issued by the competent local authorities. Incorrect disposal of waste is subject to penalties provided for by the law in force in the given area.

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