# nenob<sup>®</sup>

## Breast milk storage bags

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## EN USER MANUAL

#### 01. INTENDED USE

Bags used to store and freeze expressed breast milk.

#### 02. SAFETY

1. The breast milk bag is intended for single use only. Secondary use can cause loss of protective qualities of the bag or contamination of the food.
2. Do not thaw or heat the milk in a microwave oven or a pot with boiling water, to avoid risk of burns. High temperature can cause loss of nutritional value of milk.
3. Do not fill the bag over the 150ml line, because milk expands when freezing.
4. Do not store milk on the fridge door. Frequent opening and closing of the doors can influence the shelf life of the milk.
5. Remember to mark the expression date on the storage bags.

#### 03. HYGIENE

1. Thoroughly wash your hands with soap and warm water, and dry with a clean towel before touching the breast, storage bag and breast pump elements.
2. Remember to not touch the inner side of the bag.

#### 04. MILK STORAGE TIPS

Tips on milk storing time	Freshly expressed milk	Defrosted milk
Room temperature	4-6 hours in temperature from 19 to 26°C	1-2 hours in a temperature up to 24°C
Fridge	3-8 days in approx. 4°C or less	24 hours in 4°C
Freezer	6-12 months in a temperature from -18°C to -20°C	Do not freeze again

#### Milk storage tips

1. If you are away from home, store the bottle or milk container in a dry place.
2. If you are home, fill the storage bags and place them in the freezer right after expressing.
3. Remember to not fill the bags entirely with milk (no more than  $\frac{3}{4}$  storage bag capacity) before storing them in the freezer, because milk expands when freezing.

Always mark the bags with the expression date.

4. After finishing the expression, screw the cap tightly onto the bag.
5. To keep all the essential vitamins and minerals in the milk, defrost the storage bag in the fridge overnight, or under a stream of warm water (no more than 37°C).

#### 05. SERVICE CENTRE

Details, contact and website address can be found at: <https://nenop.pl/contact>

*Specifications and contents of the kit are subject to change without notice. We apologise for any inconvenience.*

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