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Grasso









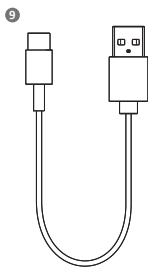
Table of contents

Spis treści


6	Instrukcja obsługi	PL
9	User manual	EN
11	Bedienungsanleitung	DE
15	Návod k použití	CZ
17	Használati utasítás	HU
20	Používateľská príručka	SK
23	Användarhandbok	SE
26	Käyttäjän käsikirja	FI
29	Brukerhåndbok	NO
31	Brugermanual	DK
34	Gebruikershandleiding	NL
37	Manual del usuario	ES
40	Manuale utente	IT
43	Manuel de l'utilisateur	FR
46	Manual de utilizare	RO

A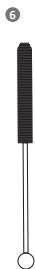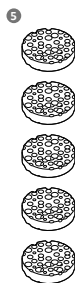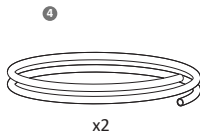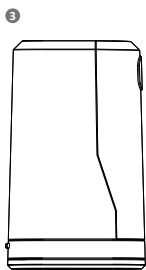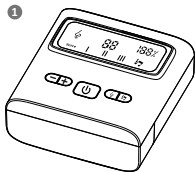

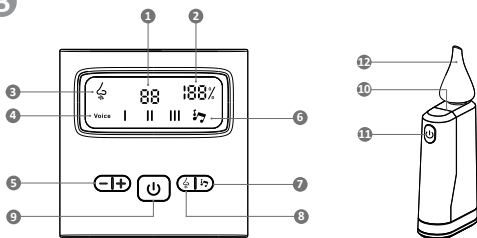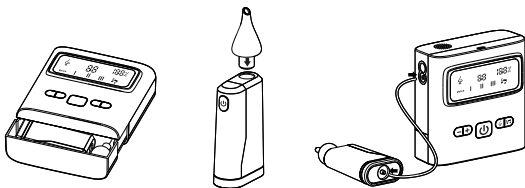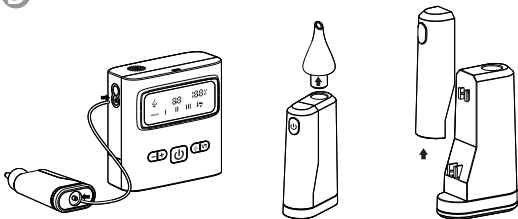B**C****D**


E

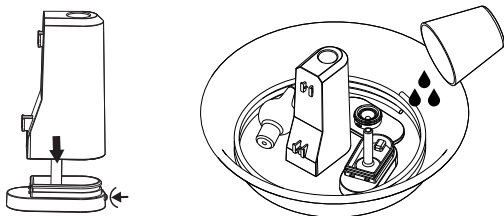F

G

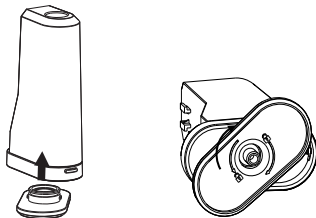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Szanowny Kliencie,

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USER MANUAL

Dear Customer,

Thank you for purchasing the Neno Grasso nasal aspirator. The product is used to suction secretions from a child's nose. The device is intended for children from birth to 6 years of age. These instructions for use contain all the information necessary for correct use. Please familiarise yourself with it and keep it in case you need to use it again.

01. PRECAUTIONS

1. Do not disassemble or repair the product yourself.
2. Disconnect the device from the power supply when it is fully charged.
3. Do not use the product if any component is damaged or missing and if the device overheats or operates erratically.
4. Do not use accessories or spare parts other than those supplied.
5. Do not store the device near water or a heat source. Do not expose the device to extreme temperatures, high humidity or direct sunlight.
6. The device must be operated by an adult, do not allow children to use the product alone.
7. Do not store the device and its packaging in a place that is easily accessible to children. Keep the cable and other accessories out of the reach of children.
8. This product is not a toy. Do not allow children to play with the device.
9. Ensure that the child is breathing properly and not holding his or her breath when aspirating secretions.
10. Do not suction the child's nasal secretions while the child is sleeping.
11. To avoid irritation, do not insert the tip too deeply into the nose.
12. Test the suction power on the back of an adult's hand before use.
13. Clean the device after each use.
14. Never block the product's vents or place it on a soft surface such as a bed or sofa where the vents may be blocked.
15. Stop using the product if you notice irritation in your child's nose. Do not use the aspirator if there is nasal bleeding, blood clotting disorders or head injuries.
16. Do not use this product on your ears, lips or eyes.

02. CONTENTS OF THE KIT**SEE FIG. A**

1. Aspirator
2. Tip x3
3. Container with light
4. Tube x2 (50 and 100cm)
5. Filter x5
6. Brush
7. Flat brush
8. Tweezers
9. Type-c cable
10. Storage pouch
11. Operating instructions

03. DESCRIPTION OF THE DEVICE**SEE FIG. B**

1. Suction power level indicator (L0 - L9)

2. Charge level indicator
3. Suction start indicator
4. Volume indicator for pre-loaded melodies
5. Increase/decrease suction power level; Increase/decrease volume of preloaded melodies
6. Launch indicator for pre-loaded melodies
7. Melody on/off/next melody
8. Switching suction on/off
9. Switching the device on/off
10. Light illuminating the inside of the nose
11. Switching on the light for illuminating the inside of the nose
12. Tip

04. OPERATION OF THE DEVICE

1. Remove the container, tube and tip from the accessory storage drawer. Connect the tube to the aspirator and the container, then place the tip on the container. **See FIG. C.** Make sure the tube tips are fitted correctly.
2. Press and hold the B9 power button to start the device.
3. To start the suction, press button B8 and adjust the suction power level using adjustment buttons B5. After starting the suction, the device starts at level 1 (L1) every time. To interrupt the suction, press button B8 again.
4. To start one of the three pre-loaded melodies, press the B7 button. A second and third press will switch to the second and third melody in turn. Use the B5 buttons to adjust the volume of the melodies. To switch off the pre-loaded melodies, press the B7 button a fourth time.
5. Press and hold the B9 power button to switch the device off. In standby mode, the device will automatically switch off after 10 minutes.
6. Charge the device using the cable provided. During charging, the charge level indicator illuminates. The device cannot be started while charging. Remember to disconnect the device from the power supply when it is fully charged.
7. Ensure that the child's head is in an upright position when extracting nasal secretions. Insert the tip into the nostril to seal it making sure the tip is not inserted too deeply.
8. To avoid eardrum pressure imbalance due to negative pressure, children should not hold their breath, close their mouth or swallow while using this product.

CAUTION: Do not use the device if the secretion exceeds the MAX level marked on the container. This can lead to damage to the device.

05. CLEANING AND STORAGE

1. The device should be cleaned after each use.
2. Disconnect the tube from the aspirator and the container. Then pull the tip out of the container and slide the piece with the light upwards. **SEE FIG. D.**
3. Press the button at the bottom of the container to open it and rinse any lingering secretions under running water. Soak the container (without the light element), tube and tip in water (up to 100 °C, no more than 10 minutes) and clean with the supplied brushes and mild detergent if necessary, then leave to dry. **SEE FIG. E.**
4. The container (without the light element), tips and tubes can also be sterilised in bottle sterilisers.
5. If you notice that the blue filter at the bottom of the container is covered with secretions, it needs to be replaced. To do this, turn the white base of the container, located at the bottom of the container, to the left and then gently pull it towards you. Replace the filter with a new one (we have an extra 5 pieces of replacement filter in the kit). **SEE FIG. F.**
6. To assemble the container after changing the filter, place the white base on the underside of the container and then turn clockwise. **SEE FIG. G.**
7. Make sure all components of the device are completely dry before reassembling or storing

in the aspirator drawer.

8. Switch off the device during cleaning.
9. Do not immerse the aspirator under water. Clean it with a clean, damp cloth.
10. When not in use, the product should be stored in a dry place.

06. PROBLEM SOLVING

1. Poor or no suction:

- Check that the tip is correctly inserted into the container.
- Check that the device is charged.
- Check that other components, such as the tube, are correctly fitted.
- Check whether the filter at the bottom of the container is contaminated by secretions. If so, replace the filter.

2. Irritation and pain in the nasal cavity:

- The aspirator cannot be used if there is a nose bleed, blood clotting disorders or head injuries.
- Do not insert the tip too deeply into the nasal cavity and do not press the tip too hard against the inner walls of the nasal cavity.
- Make sure that the suction power setting is not too high.

3. Difficulty in clearing nasal secretions:

- Ensure that the baby's head is in an upright position when extracting nasal secretions.
- If necessary, spray the inside of the nasal cavity with a saline solution to dilute any lingering secretions and make suctioning easier.

07. SPECIFICATION

Input: 5V 1A

Rechargeable battery: Lithium polymer 3.7V, 1500mAh

Suction force: max 60kPa \pm 5kPa

Air flow rate: 7l/min

Container capacity: 35ml

Tip material: silicone

Sound level: \leq 65dB

Charging time: approx. 2h

Operating time: up to 4h

Dimensions: 11.4x11.4x5cm

Weight: 375g

08. WARRANTY CARD

The product comes with a 24-month warranty. Warranty terms and conditions can be found at:

<https://neno.pl/gwarancja>

Details, contact and service address can be found at: <https://neno.pl/kontakt>

Specifications and contents are subject to change without notice. We apologise for any inconvenience.

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EN

The crossed out trash can symbol indicates that unusable electrical or electronic devices, its accessories (such as power supplies, cords) or components (for example batteries, if included) cannot be disposed of alongside with household waste. In order to dispose of the devices or its components (for example, batteries) deliver the device to the collection point, where it will be accepted free of charge. Disposal is subject to the recast version of the WEEE Directive (2012/19/ EU) and the Directive on batteries and accumulators (2006/66 / EC). Proper disposal of the device prevents degradation of the natural environment. Information about the collection points of the facilities is issued by the competent local authorities. Incorrect disposal of waste is subject to penalties provided for by the law in force in the given area.

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