NENO SENSO

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Postpartum washing peri bottle, 500 ml

Dear Customer,

The NENO SENSO product you have purchased is a postpartum washing peri bottle with a capacity of 500 ml. With its gentle stream, ergonomic design and simple use, it makes perineal care easier when using the toilet. It will prove useful as an additional support in maintaining daily hygiene, especially during pregnancy and after childbirth, after gynaecological operations, during menstruation or intimate infections, and also when access to a shower is difficult.

Read the enclosed instructions before use.

01. CONTENTS OF THE KIT

1 x bottle; 1 x bag; 1 x instructions;

02. INSTRUCTIONS FOR USE

1. Check the condition of the product before each use. If there are signs of damage, replace the product.
2. Unscrew the bottle and fill it with warm, not hot, water. Water that is too hot can cause burns.
3. Close the bottle and turn it upside down.
4. Point the nozzle towards the perineum and aim at the area you want to wash.

5. Press the bottle and wash the desired area.
6. After use, disassemble the bottle and wash it thoroughly in warm, soapy water. NOTE: Clean the bottle and nozzle regularly to avoid the growth of harmful bacteria.
7. Allow the elements to dry naturally. NOTE: the bottle is not water-proof! Store the dry bottle in the bag provided, in a dry, cool place.

ATTENTION: The bottle is for external use only! Do not use it for irrigation! Use the bottle only as described in the instructions! The product is not a toy!

03. SPECIFICATION

Capacity: 500 ml • **Bottle material:** PP + FEP • **Neck material:** PP • **Bag material:** non-woven • **Bag size:** 31 x 16 cm • **Bottle dimensions:** overall: 34 cm; folded: 22.5 cm; diameter: 7 cm

04. SERVICE

Details, contact and service address can be found at:

<http://neno.pl/kontakt>

Specifications and contents are subject to change without notice.

We apologise for any inconvenience.

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The logo for 'neno' is written in a bold, rounded, teal-colored font. A small registered trademark symbol (®) is located at the top right of the letter 'o'.**Producător:**