# neno<sup>®</sup>

## Vento Twin

## Table of contents

### Spis treści


<b>6</b>	Instrukcja obsługi	<b>PL</b>
<b>10</b>	User manual	<b>EN</b>
<b>14</b>	Bedienungsanleitung	<b>DE</b>
<b>18</b>	Návod k použití	<b>CZ</b>
<b>22</b>	Használati utasítás	<b>HU</b>
<b>26</b>	Používateľská príručka	<b>SK</b>
<b>30</b>	Användarhandbok	<b>SE</b>
<b>34</b>	Käyttäjän käsikirja	<b>FI</b>
<b>38</b>	Brukerhåndbok	<b>NO</b>
<b>42</b>	Brugermanual	<b>DK</b>
<b>46</b>	Gebruikershandleiding	<b>NL</b>
<b>50</b>	Manual del usuario	<b>ES</b>
<b>54</b>	Manuale utente	<b>IT</b>
<b>59</b>	Manuel de l'utilisateur	<b>FR</b>
<b>63</b>	Manual de utilizare	<b>RO</b>

A


1

2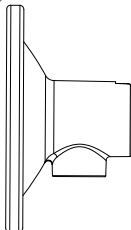


3

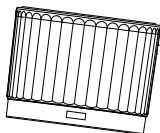4

6

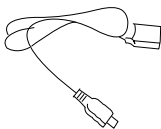
7

5








8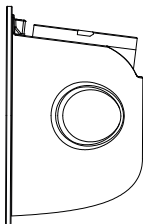

9

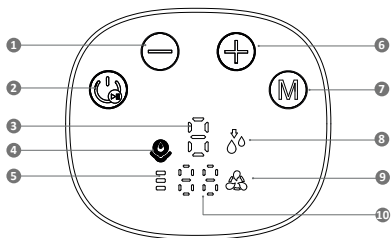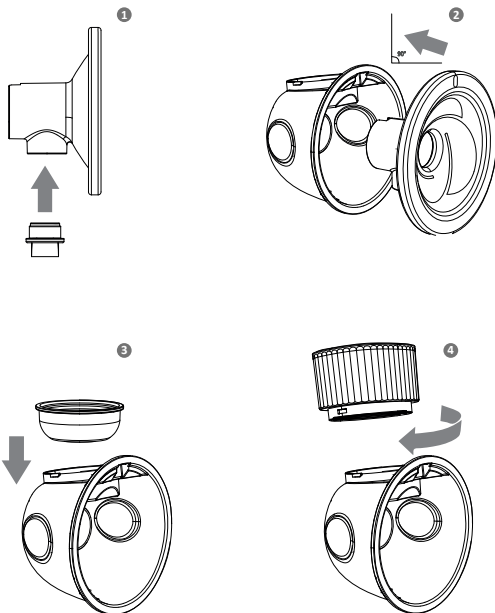**B****C**

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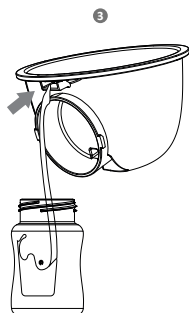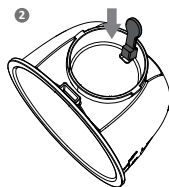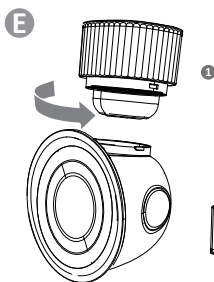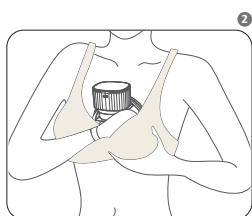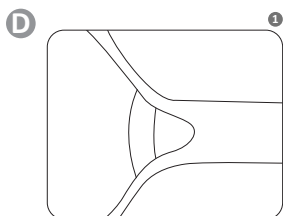Szanowny Kliencie,

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**USER MANUAL**

Dear Customer,

The Neno Vento Twin device you have purchased is an electronic, three-phase shell breast pump for expressing breast milk from a woman's breast after childbirth. Read the instruction manual before use.

**01. PRECAUTIONS**

- Do not use a breast pump during pregnancy - it can cause premature labour.
- Do not use the machine if you have a chronic illness or hypersensitivity that would cause you pain when using the breast pump. If using the device causes you pain or severe discomfort, switch it off and move the breast pump away from the breast.
- Do not place the product where it is in danger of falling into sinks, bathtubs or other bodies of water.
- Do not use the device in the bath or shower.
- Do not place the device under water or in other liquids.
- Do not wash the device under running water.
- The breast pump and its components are not heat resistant - keep the device away from hot surfaces and open flames.
- Do not use the device near flammable materials.
- Keep the silicone parts of the breast pump away from hot surfaces to avoid distortion.
- Always unplug the device after charging if you do not need to recharge the device's battery.
- The breast pump should not be left unattended when connected to a power source.
- Never use the device if it has been dropped or otherwise damaged.
- Do not use a device whose power cord or plug is damaged.
- Do not connect the device to a power socket that has been in contact with water or other liquids.
- If the device has been exposed to water or other liquids - do not touch the device directly, disconnect it from the power supply, switch it off and contact the manufacturer.
- Do not connect the charging device to a voltage higher than specified.
- This device is intended for use by a single user. Use of the breast pump by more than one person carries a risk of harm to the users and voids the warranty.
- Do not repair or modify the device yourself.
- Use the breast pump only as described in the instructions. Do not use the device for any other purpose.
- Inspect all components of the breast pump before each use. Do not use the breast pump if any component is visibly damaged.
- Keep the device out of the reach of children and pets.
- Do not clean machine parts with abrasive agents.
- Do not heat the discharged milk in the microwave. Milk heated in a microwave can burn the baby's face because heating with microwave waves can create high temperature bubbles in the liquid. In addition, microwave waves can change the composition of the milk.
- Do not bring the milk to the boil while heating it.
- Take special care when using the breast pump or its accessories near children.

**02. BEFORE FIRST USE**

- Connect the breast pump to the power supply. Charge the batteries for 2 hours.
- Remove the breast pump.
- Disconnect the parts of the breast pump that will come into direct contact with the breast or milk: pull the diaphragm and the breast shield with the connector out of the milk container, then pull the valve down to disconnect it from the connector.
- Clean the device components (refer to the CLEANING AND DISINFECTION section).

**NOTE!**

Use the breast pump in an upright position. Using in any other position (e.g. lying down) will lead to the breast pump not fitting properly to the breast and milk spilling out.

### 03. ELEMENTS

#### FIG. A

1. 2 x Breast shield lid
2. 2 x Breast shield 25 mm with connector
3. 2 x Pumping device
4. 4 x Diaphragm
5. 4 x Stopper
6. 2 x Charging cable
7. 4 x Valve
8. 2 x Milk container
9. 2 x Breast shield reducers: 15mm / 17mm / 19mm / 22mm

### 04. DISPLAY ICONS

#### FIG. B

1. Changing the suction level - Press „-“ to reduce the suction power.
2. Power / pause button - Hold the button for 2 seconds to turn the machine on or off. When the device is on press to pause the breast pump.
3. Selected suction level icon - Shows the currently selected suction power level (L1-L9).
4. Stimulation mode icon.
5. Battery status - The battery icon shows the state of charge of the battery. Connect the device to charge when the battery is close to empty.
6. Changing the suction level - Press „+“ to increase the suction power.
7. Mode change button - Press the button to change the mode of the device. There are 3 modes available (stimulation, deep suction, mixed).
8. Deep extraction mode icon.
9. Mixed mode icon.
10. Session duration indicator - Shows the duration of the session. The pump will automatically switch off after 30 minutes of operation.

### 05. CLEANING AND DISINFECTION

Washing and disinfecting are two different operations. They must be carried out separately to protect your baby and to ensure that the breast pump works properly.

Washing - washing the surface of the device and removing dirt.

Disinfection - killing organisms such as bacteria or viruses that may be present on the surface of the equipment.

**Breast shield with connector, diaphragm, valve, reducers, stopper, milk container:**

Wash and disinfect before first use.

Wash after each use.

Once a day - disinfection.

#### A. WASHING:

- Disassemble all parts of the breast pump (except the pumping device) and wash in warm soapy water for 5 minutes, then rinse. Do this immediately after using the machine. This will remove milk deposits and prevent bacterial growth.
- Only use drinkable tap water for washing. Use bottled water if the water in your tap does not meet this requirement.
- When cleaning silicone parts, do not use brushes. Be careful not to pierce them through.
- Use a mild dishwashing liquid or soap.
- Allow the parts to dry naturally. Store dry device's parts in a cool, dry place when you are not using them.

#### B. DISINFECTION:

Diaphragm, valve, reducers, breast shield with connector, stopper, milk container

- Fill the pot or steriliser with water so that all the components inside (diaphragm, valve, regulators, breast shield with connector, stopper and milk container) are covered when placed inside. Bring to the boil. Place the breast pump components in boiling water for 10 minutes. Do not boil. It may cause damage to the components.
- Allow the items to dry naturally. Store dry kit items in a cool, dry place when you are not using them.

### C. CLEANING THE PUMPING DEVICE:

- Wipe the pumping device with a soft, wet cloth and a mild dishwashing liquid/soap. Other cleaners or abrasives may damage the device and its display.
- When not using the pumping device, store it in a cool, dry place.

## 06. BREAST PUMP ASSEMBLY

### FIG. C

Attach the components as shown in **FIG. C.1-4**

**NOTE:** Wash your hands thoroughly with soap and water before touching the breast pump and breast. Make sure all parts of the kit are clean and dry before each use.

1. Connect the breast shield to the connector in the direction shown **FIG. C.1**.
2. Attach the breast shield with the connector to the milk container starting by placing the connector in the groove. Make sure the breast shield is in the correct position **FIG. C.2**.
3. Place the membrane in the milk container **FIG. C.3**.
4. Attach the pumping device to the top of the milk container by pressing down the membrane. Then press the device down and tighten clockwise **FIG. C.4**. Make sure it is in the correct position.

Councils:

- Before each use, check the parts of the breast pump for damage. If necessary, replace damaged parts with new ones.
- Always check the cleanliness of components before using the device.
- If the diameter of the breast shield is too large, use the supplied reducer by inserting it inside the breast shield.

## 07. USE OF A BREAST PUMP

### FIG. D

1. Place the breast shield on your breast so that the nipple is inside the drainage channel. Press the breast shield and breast pump connector against your breast using your thumb and index finger. **FIG. D.1**
2. Place the breast pump in the bra/lactation girdle. **FIG. D.2**. Turn on the breast pump by pressing the power button and select the mode of operation that suits you.

### NOTES:

The breast pump starts in stimulation mode. Set the level of suction power that is comfortable for you using the +/- buttons.

- Stimulation mode - rapid pumping that mimics your baby's natural sucking of breast milk to stimulate milk flow. After 2 minutes, the breast pump will automatically activate the deep pumping mode. Adjust the suction level using the +/- buttons to find the highest level of suction that is comfortable for you. If milk starts to flow from your breast before two minutes have elapsed, press the mode change button to manually switch to deep extraction mode.
- Deep pumping mode - slower pumping that gently and efficiently sucks milk from the breast. In intake mode, increase the suction until you feel slight discomfort (not pain) and reduce the suction by one degree.
- Mixed mode - mimics the baby's sucking reflex. Can be used after pumping to help empty the breast and increase lactation.

## 08. TRANSFERRING MILK

### FIG. E

1. Hold the breast pump upright and carefully unscrew the pump counterclockwise **FIG. E.1**. Remove the diaphragm. Holding the container upright, insert the silicone plug into the hole located under the diaphragm as shown in **FIG. E.2**.
2. Hold the milk container horizontally with the breast shield facing upwards - you can lay it on a flat surface (do not put the container in any other position to prevent the milk from flowing out)
3. Carefully transfer the milk into the bottle/storage container as shown in **FIG. E.3**.

Remove and clean all breast pump components after use.

## 09. CHARGING THE BREAST PUMP

Fully charge the breast pump battery before putting it into storage if you intend not to use the device for a long period of time. Charge the device battery before it is completely depleted. This extends the life of the battery. If the breast pump has been stored in a hot place, the device battery may not work at first. Allow the breast pump to cool down for one hour before you start using it.

Indications:

- Full blinking battery - low battery, connect device to charge.
- Incomplete blinking battery - battery connected to charge, current battery charge status.
- Full battery - battery fully charged.

#### 10. REPLACEMENT OF CONSUMABLES

Consumable components are subject to natural wear and tear during operation of the breast pump and are therefore not covered by the manufacturer's warranty.

Worn components of the breast pump that come into contact with milk should be replaced. A worn component has a crack, deformation, tarnishing or a distinct change in colour.

- The diaphragm should be replaced every 3-4 months (if you use the breast pump 1-3 times a day) or every 6-8 weeks (if you use the breast pump more than 3 times a day).
- The valve should be replaced every 3-4 months (if you use the breast pump 1-3 times a day) or every 6-8 weeks (if you use the breast pump more than 3 times a day).
- The breast shield and connector should be replaced every 6 months if you use the breast pump more than 3 times a day.

#### 11. LONG-TERM STORAGE

The breast pump has a built-in rechargeable battery. To protect itself from damage, the device enters storage mode after a long period of inactivity. To deactivate the storage mode, connect the device to power and charge it for 2 hours. Before you put the breast pump away for a long period of time, make sure that:

- All breast pump components are clean and dry.
- You have cleaned the pumping device.
- The device's battery is fully charged.

#### 12. FAQ

The device does not start: make sure the breast pump is plugged into a working electrical outlet. If the breast pump has no power source, its display will not turn on and no error message appears.

No/poor suction: make sure all breast pump parts are positioned correctly. Take the breast pump apart and make sure all parts are clean and undamaged. Reassemble the breast pump according to the instructions in the LACTATOR ASSEMBLY section. When pumping, make sure the breast shield is fully seated against the breast.

More milk has flowed from the breast than the container can hold: use a damp (not wet) cloth to wipe the breast pump. Wash the parts of the breast pump that hold milk according to the instructions in the CLEANING AND DISINFECTING section. When all parts of the machine are dry, reassemble.

#### 13. SPECIFICATION

**Milk container volume:** 150ml

**Battery capacity:** 3.7V 1100mAh Li-ion

**Input power:** 110-240V

**Output power:** 5V 1A

**Operating time:** up to 120 min

**Charging time:** approximately 120 min

**Dimensions of one pump:** 7.0 x 6.3 x 4.8 cm

**Weight of one pump:** 143 g

Suction power:

Stimulation mode (9 levels): 5-21 kPa

Deep extraction mode (9 levels): 14-42 kPa

Mixed mode (9 levels): 15-40 kPa

Permissible variation  $\pm 5$  Kpa.

#### 14. WARRANTY CARD / COMPLAINTS

The product comes with a 24-month warranty. Warranty terms and conditions can be found at:

<https://nenop.pl/gwarancja>

Details, contact and service address can be found at: <https://nenop.pl/kontakt>

Specifications and contents are subject to change without notice. We apologise for any inconvenience.

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EN

The crossed out trash can symbol indicates that unusable electrical or electronic devices, its accessories (such as power supplies, cords) or components (for example batteries, if included) cannot be disposed of alongside with household waste. In order to dispose of the devices or its components (for example, batteries) deliver the device to the collection point, where it will be accepted free of charge. Disposal is subject to the recast version of the WEEE Directive (2012/19/ EU) and the Directive on batteries and accumulators (2006/66 / EC). Proper disposal of the device prevents degradation of the natural environment. Information about the collection points of the facilities is issued by the competent local authorities. Incorrect disposal of waste is subject to penalties provided for by the law in force in the given area.

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