nenò[®]

Compresso

Table of contents

Spis treści

3	Instrukcja obsługi	PL
6	User manual	EN
9	Bedienungsanleitung	DE
13	Návod k použití	CZ
16	Használati utasítás	HU
19	Používateľská príručka	SK
22	Användarhandbok	SE
25	Käyttäjän käsikirja	FI
28	Brukerhåndbok	NO
31	Brugermanual	DK
34	Gebruikershandleiding	NL
37	Manual del usuario	ES
40	Manuale utente	IT
44	Manual de utilizare	RO
47	Korisnički priručnik	HR/BA
50	Uputstvo za upotrebu	RS/ME
53	Uporabniški pravilnik	SI
56	Εγχειρίδιο χρήστη	GR
59	Lietotāja rokasgrāmata	LV
62	Vartotojo vadovas	LT
65	Kasutusjuhend	ET
68	Посібник користувача	UA
72	Ръководство за потребителя	BG
75	Упатство за употреба	MK
78	Manual do utilizador	PT
82	Manuel de l'utilisateur	FR

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USER MANUAL

NENO COMPRESSO

Thank you for purchasing the NENO COMPRESSO hot/cold pack model CS-CH-III 110g. The set contains two compresses for heat therapy (for adults) or cold therapy (for adults and children). Please read the instructions below before use.

01. WARNINGS AND PRECAUTIONS

1. The product is intended for external use.
2. Check the temperature of the wrap before use.
3. Do not use the product heated for longer than recommended. Excessive heating or prolonged contact with the skin may cause skin damage, including burns.
4. When heated, the compress should be warm to the touch, but not hot.
5. Do not use in trauma, sprains when inflammatory exudate still persists after 24 hours.
6. Do not use if the skin is unable to respond properly to temperature (e.g. during local anaesthesia or when taking analgesics).

7. Do not use in infants.
8. Do not use for injuries with inflammation.
9. Do not apply directly to skin or damaged skin with bleeding.
10. The product is not a toy, keep away from children.
11. Clean with mild soap and water.
12. It is normal for the wrap to expand slightly when heated for the recommended time.
13. If the compress is heated for longer than the recommended time and it breaks or inflates heavily, switch off the microwave oven immediately. Then wait for it to cool down before discarding.
14. If in doubt or if the packaging shows any signs of deformation, it should be discarded. Packaging that has been overheated should never be reused.
15. If you have circulation problems, consult your doctor before use.
16. Do not fall asleep while applying the compress.
17. Do not pierce the packaging with a sharp object. Do not deliberately press with a heavy object to damage. Do not hit the product or swallow the contents.
18. In case of extreme, prolonged or chronic pain, contact your doctor.
19. If the compress is damaged, torn or ruptured, discontinue use and dispose of the contents in the bin.
20. Avoid contact with eyes and skin if the packaging breaks. In case of contact, flush with clean water. If the packaging is punctured or opened and the contents accidentally swallowed, drink plenty of water.
21. Store in a dry, well-ventilated and cool place.
22. Do not place the cover in the microwave oven.

02. PURPOSE OF THE PRODUCT

Heat therapy can provide relief from uterine contractions, which occur when the organ returns to its original size.

In the case of perineal pain, applying **cold therapy** to the painful area may help reduce swelling and relieve pain.

NOTE: A single use should not last longer than 20 minutes, with a break of at least one hour before the next use, unless otherwise advised by a doctor.

03. INSTRUCTIONS FOR USE

A) Heating in the microwave:

Place one piece in the microwave oven flat.

NOTE: before first use, place the compress in the microwave for half the time listed below. Check that the temperature is suitable. Extend the process if necessary. The heating time given is a reference time. The time may vary and should be adjusted according to the power and parameters of the individual microwave oven.

700 WAT - 25

1000 WAT - 19 seconds

1250 WAT - 16 seconds

Each compress should be heated separately. Check the temperature of the compress each time before use. If necessary, heat the compress for an additional 5 seconds.

B) Heating in a water bath:

Fill a large vessel with enough water to completely submerge the compress without touching the bottom of the vessel. Bring the water to the boil and remove the vessel from the heating surface. **Do not immerse the compress while the water is being heated.** Use tongs, for example, to dip the compress in and out of the water.

Do not leave the compress unattended while it is heating.

Dip the compress into the water and leave it for 2 minutes. If the compress has not reached the desired temperature, immerse it again (in the same water) for an additional minute. Avoid prolonged contact between the compress and the pot or heat source, as excessive contact can lead to damage to the product. Dry the compress with a towel, for example, before applying it to the body. You can also use it wrapped in a towel if the temperature is too high.

NOTE: Before each use, the product must be checked for signs of damage or leakage. If damage is found, discontinue use.

Check the temperature of the compress before each use and allow to cool if necessary.

C) Cooling in the freezer

Place the compress in the freezer for approximately 4 hours. Make sure the temperature is right before use.

Once heated or cooled, place the compress in the cover. The covers are designed for single use. The compress can be used both with and without the cover.

NOTE: Do not heat the compress out of the freezer and do not put the warm compress in the freezer. The product should be at room temperature. There is a risk of damaging the product.

NOTE: Always check the temperature of the compress before application. For external use only. After each use, the compress should be washed, dried and stored in a cool place.

04. CLEANING AND DISINFECTION

Washing and disinfection are two different activities that must be carried out separately.

Washing - wash the surface of the product and remove dirt. Use only tap water and a mild liquid or soap for washing. Dry the compress with a towel, for example.

Disinfection - killing organisms such as bacteria or viruses that may be present on the surface of the device. For disinfection, it is recommended to use alcohol-free agents e.g. Cleanisept® Wipes. Do not use alcohol-based disinfectants as these may damage the surface of the compress.

05. EXPLANATION OF SYMBOLS

SEE TABLE 1

1. CE conformity marking
2. Medical device
3. Producer
4. Authorised representative in the European Community
5. Importer
6. Unique identification code of the product

7. Production batch number
8. Expiry date
9. Read the instructions for use
10. Keep out of direct sunlight
11. Protect from moisture
12. Flat paperboard

06. CONTENTS OF THE KIT

2 x gel compress, 16 x pouch, instruction manual

07. SPECIFICATION

COMPRESS: dimensions: 24x7 cm, weight 110g, INGREDIENTS: water, glycerol, sodium polyacrylate; outer material: PVC

COVER: Dimensions: 25.5x8 cm, material: non-woven,

SHELF LIFE:

3 years from date of manufacture

08. WARRANTY CARD

Details, contact and service address can be found at: <https://neno.pl/kontakt>

Specifications and contents are subject to change without notice. We apologise for any inconvenience.

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Shanghai International Holding Corp.Gmbh (Europe)
Shanghai Chuangshi Medical Technology (Group) Co., Ltd.