1.	2.	3.	4.	5.	6.	7.	8.
9.	10.	11.	12.	13.	14.	15.	16.

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NENO FIORE-Hydrogel cooling patch with lanoline

Dear Customer,

Thank you for choosing Neno Fiore lanolin hydrogel breast pads. This product gently cools and soothes sore nipples, restores its natural moisture and protects against chafing. Please read the instructions below before use.

01. WARNINGS

- Use as intended.
- Store in a dry and cool place, out of the reach of children. The product is not a toy.
- **Do not use on bleeding warts. The liner is not designed to treat deep wounds.**
- If an allergic reaction, irritation, burning or bleeding of the nipples occurs, discontinue use and consult a specialist.
- Do not use if the sachet is damaged or open. Do not use the product if the smell, consistency or colour has changed.
- If the pad becomes soiled, stop using it - cleaning may adversely affect performance.
- A serious incident involving a device must be reported to the producer and to the competent authority of the Member State in which the user or patient is resident.

02. INTENDED USE AND INSTRUCTIONS FOR USE

This product is intended for external use to reduce temperature through physiotherapy.

- Wash your hands thoroughly. Open the sachet, take out the insert and remove the protective film.
- Gently apply the insert directly to the nipple, avoiding touching the hydrogel layer.
- Before feeding, remove the breast pad and place on a clean surface, gel side up. Wash the breast with warm water and pat dry.
- To increase the cooling effect, the unopened sachets can be placed in the refrigerator for a short time before use.

03. EXPLANATION OF SYMBOLS

- | | |
|--|--|
| 1. CE conformity marking | 9. Read the instructions for use |
| 2. Medical device | 10. Do not reuse |
| 3. Manufacturer | 11. Do not use if the packaging is damaged |
| 4. Authorised representative in the European Community | 12. Keep out of direct sunlight |
| 5. Importer | 13. Protect from moisture |
| 6. Unique identification code of the product | 14. Warning |
| 7. Production batch number | 15. Flat paperboard |
| 8. Expiry date | 16. Keep out of reach of children |

04. TECHNICAL DATA

Kit contents: 2 pcs. per sachet, 5 sachets | **Pad dimensions:** 7.5 x 7.3 cm | **Model:** CSI-CP-I 7.5X7.3 cm | **Ingredients:** water, glycerine, sodium polyacrylate, lanolin | **Expiry date:** 2 years from date of producer

05. CARD

Details, contact and service address can be found at: <https://neno.pl/kontakt>

Technical data and contents are subject to change without notice. We apologise for any inconvenience.

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Shanghai International Holding Corp.GmbH (Europe)
Shanghai Chuangshi Medical Technology (Group) Co., Ltd.