neno[®]

Corto







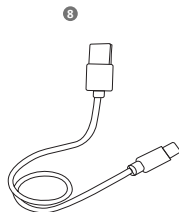
Table of contents

Spis treści


6	Instrukcja obsługi	PL
9	User manual	EN
11	Bedienungsanleitung	DE
14	Návod k použití	CZ
17	Használati utasítás	HU
19	Používateľská príručka	SK
22	Användarhandbok	SE
25	Käyttäjän käsikirja	FI
27	Brukerhåndbok	NO
30	Brugermanual	DK
32	Gebruikershandleiding	NL
35	Manual del usuario	ES
38	Manuale utente	IT
41	Manuel de l'utilisateur	FR
43	Manual de utilizare	RO
46	Korisnički priručnik	HR/BA
49	Uputstvo za upotrebu	RS/ME
51	Uporabniški pravilnik	SI
54	Εγχειρίδιο χρήστη	GR
57	Lietotāja rokasgrāmata	LV
59	Vartotojo vadovas	LT
62	Kasutusjuhend	ET
65	Посібник користувача	UA
68	Ръководство за потребителя	BG
71	Упатство за употреба	MK
74	Manual do utilizador	PT

A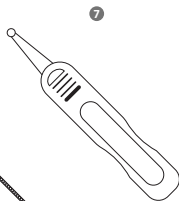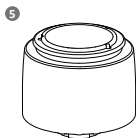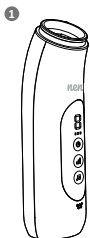


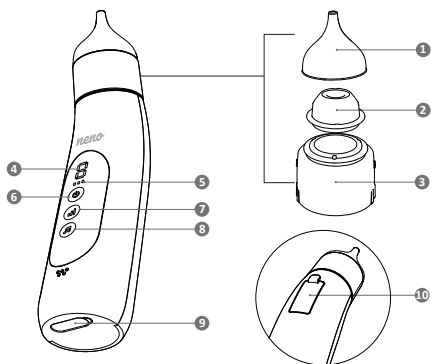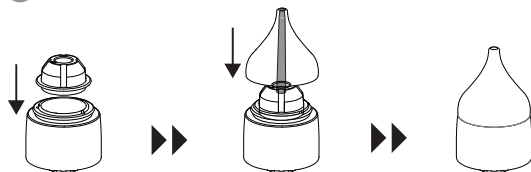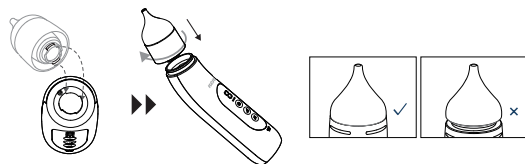B**C****D**


E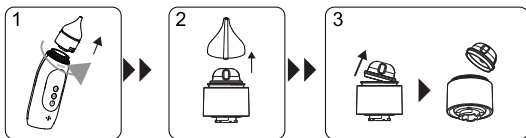

F

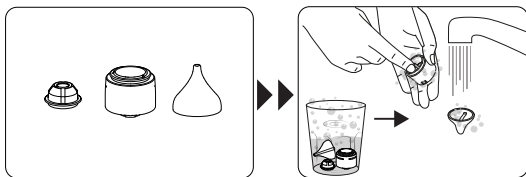G

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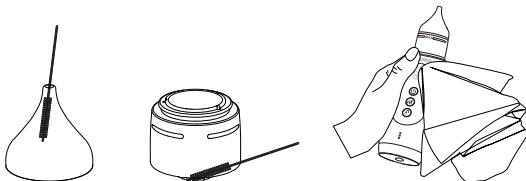Szanowny Kliencie,

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USER MANUAL

Dear Customer,

Thank you for purchasing the Neno Corto nasal aspirator. This product is designed to remove secretions from your child's nose. It is suitable for children from birth to 6 years of age. This user manual contains all the information necessary for proper use. Please read it carefully and keep it for future reference.

01. PRECAUTIONS

1. Do not disassemble or repair the product yourself.
2. Disconnect the device from the power supply once it is fully charged.
3. Do not use the product if any part is damaged or missing, or if the device overheats or malfunctions.
4. Do not use accessories or spare parts other than those included in the kit.
5. Do not store the device near water or heat sources. Avoid exposing the device to extreme temperatures, high humidity or direct sunlight.
6. The device must be operated by an adult; do not allow children to use the product on their own.
7. Do not store the device or its packaging in a place easily accessible to children. Keep the cable and other accessories out of the reach of children.
8. This product is not a toy. Do not allow children to play with the device.
9. Ensure that the child is breathing normally and not holding their breath while suctioning.
10. Do not suction secretions from your child's nose while they are asleep.
11. To avoid irritation, do not insert the tip too deeply into the nose.
12. Test the suction power on the back of an adult's hand before use.
13. Clean the device after each use.
14. Stop using the device if you notice irritation in your child's nose. Do not use the aspirator in case of nosebleeds, blood clotting disorders or head injuries.
15. Do not use this product on the ears, mouth or eyes.

02. CONTENTS OF THE SET**SEE FIG. A**

1. Aspirator
2. Tips
3. Container lid
4. Secretion container
5. Brush
6. Tweezers
7. Type-C cable
8. Storage bag
9. User manual

03. DESCRIPTION OF THE DEVICE**SEE FIG. B**

1. Tip
2. Container lid
3. Secretion container
4. Suction power level indicator
5. Charge level indicator

6. Power button (on/off)
7. Increasing/decreasing suction power level
8. Turning the melody on/off/next melody
9. USB-C charging port
10. Buffer tank

04. DEVICE OPERATION

1. Place the lid on the container and then attach one of the tips. Ensure that the tip is correctly fitted. **SEE FIG. C.** Then place the container in the aspirator base and turn it to lock it in place. **SEE FIG. D.**
2. Press and hold the power button B6 to start the device. Then use the B7 button to set one of five suction levels. The device has a memory function – when restarted, it will start operating at the last suction level used.
3. Briefly pressing button B6 while the device is running will pause suction; pressing it again will restart it.
4. To play one of the three pre-loaded melodies, press and hold the B8 button. Pressing it a second and third time will switch to the second and third melodies, respectively. To turn off the pre-loaded melodies, press and hold the B8 button.
5. Press and hold the B6 power button to turn off the device.
6. The aspirator is equipped with two soft tips that can be adjusted to suit the age and shape of your child's nose – however, remember not to insert the tip too deeply to ensure your baby's comfort and safety.
7. Do not exceed the level of secretions above the tube inside the reservoir to prevent fluid from entering other parts of the device.
8. The aspirator is equipped with a buffer reservoir (B10), which provides additional protection – if the level of secretions is too high, the excess is automatically drained through an internal tube into this reservoir, protecting the device from contamination and damage.
9. Charge the device using the cable included in the set. The charge level indicator lights up during charging. Do not use the device while charging. Remember to disconnect the device from the power supply when it is fully charged.
10. Ensure that your child's head is in an upright position when removing nasal secretions. Insert the tip gently into the nostril to create a seal, without inserting it too deeply.
11. To avoid eardrum pressure imbalance caused by negative pressure, children should not hold their breath, close their mouths or swallow while using this product.

05. CLEANING AND STORAGE

1. The device should be cleaned after each use.
2. Turn the container upside down and pull it upwards. Remove the tip by pulling it upwards, then remove the lid from the container. **SEE FIG. E.**
3. Rinse out any remaining secretions under running water. Immerse the container, lid and tip in water (at a temperature of up to 100 °C, for no longer than 10 minutes) and clean with a mild detergent if necessary. **SEE FIG. F.** Clean hard-to-reach areas with the brush included in the set, then leave the parts to dry. **SEE FIG. G.**
4. The container, lid and tips can also be sterilised in bottle sterilisers.
5. Before reassembling or storing the device in the bag, make sure that all parts of the device are completely dry.
6. Switch off the device during cleaning.
7. Do not immerse the aspirator in water. Clean it with a clean, damp cloth.
8. When not in use, store the product in a dry place.

06. TROUBLESHOOTING

1. Weak or no suction:
 - Check that the tip is correctly inserted into the container.

- Check that the device is charged.
 - Check that other components, such as the container lid, are correctly fitted.
2. Irritation or pain in the nasal cavity:
 - Do not use the aspirator in case of nosebleeds, blood clotting disorders or head injuries.
 - Do not insert the tip too deeply into the nasal cavity and do not press the tip too hard against the inner walls of the nasal cavity.
 - Ensure that the suction level is not set too high.
 3. Difficulties in removing nasal secretions:
 - Ensure that the child's head is in an upright position when suctioning nasal secretions.
 - If needed, apply a saline spray the inside of the nasal cavity with saline solution to thin the accumulated secretions and facilitate suction.

07. SPECIFICATIONS

Input: 5V 1A

Battery: 3.7V lithium polymer, 1000mAh

Suction power: max 60kPa $\pm$ 5kPa

Air flow: 3l/min

Container capacity: 10 ml

Tip material: silicone

Sound level: $\leq$ 60dB

Charging time: approx. 2 hours

Device operating time: up to 3 hours

Dimensions: 17.6 x 4.4 x 5.2 cm

Weight: 160g

08. WARRANTY CARD

The product is covered by a 24-month warranty. The warranty terms and conditions can be found at: <https://neno.pl/gwarancja>.

Details, contact information and service address can be found at: <https://neno.pl/kontakt>.

Specifications and kit contents are subject to change without notice. We apologise for any inconvenience.

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