nenor[®]

Ovio

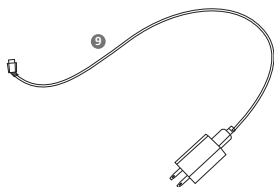
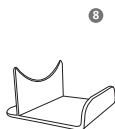
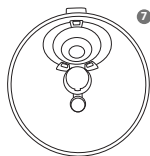
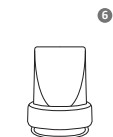
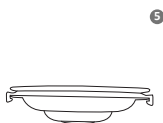
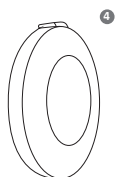
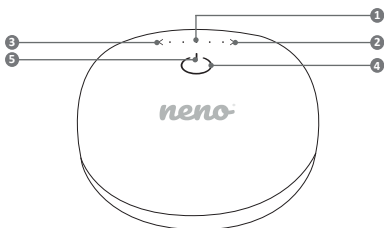
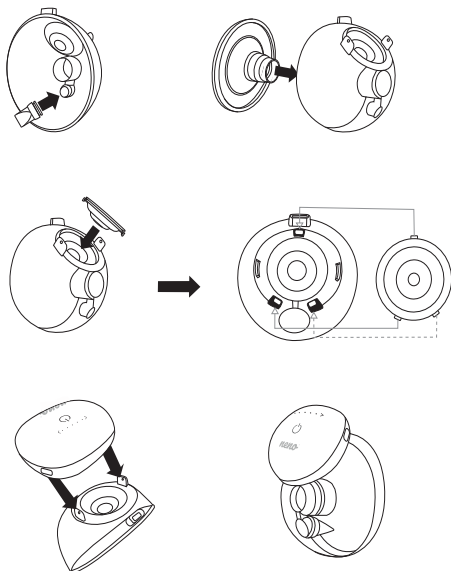
Table of contents

Spis treści

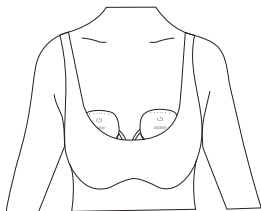
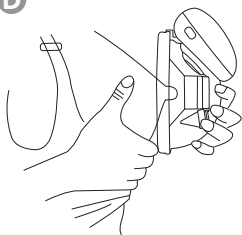
6	Instrukcja obsługi	PL
11	User manual	EN
17	Bedienungsanleitung	DE
23	Návod k použití	CZ
29	Használati utasítás	HU
35	Používateľská príručka	SK
40	Användarhandbok	SE
46	Käyttäjän käsikirja	FI
52	Brukerhåndbok	NO
57	Brugermanual	DK
63	Gebruikershandleiding	NL
69	Manual del usuario	ES
75	Manuale utente	IT
81	Manuel de l'utilisateur	FR
87	Manual de utilizare	RO
93	Korisnički priručnik	HR/BA
98	Uputstvo za upotrebu	RS/ME
104	Uporabniški pravilnik	SI
110	Εγχειρίδιο χρήστη	GR
116	Lietotāja rokasgrāmata	LV
122	Vartotojo vadovas	LT
127	Kasutusjuhend	ET
133	Посібник користувача	UA
139	Ръководство за потребителя	BG
145	Упатство за употреба	MK
151	Manual do utilizador	PT

A

B**C**

D

E

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USER MANUAL

Dear Customer,

The Neno OVIC device you have purchased is an electronic, three-phase shell breast pump with a dedicated mobile app. The device is used to express milk from a woman's breast after childbirth and has been designed with comfort, effectiveness and discretion in mind. The mobile app allows you to remo-

tely control the breast pump, monitor sessions and adjust settings to suit your individual needs. This user manual contains all the information you need to use the device correctly. Please read it carefully and keep it for future reference.

01. PRECAUTIONS

- Do not use the breast pump during pregnancy – this may cause premature labour.
- Do not use the device if you suffer from chronic illnesses or hypersensitivity that would cause pain when using the breast pump. If using the device causes pain or severe discomfort, turn it off and remove the breast pump from your breast.
- Do not place the product in places where it could fall into a sink, bathtub or other water containers.
- Do not use the device in the bath or shower.
- Do not place the device under water or in other liquids.
- Do not wash the device under running water.
- The breast pump and its components are not resistant to high temperatures – keep the device away from hot surfaces and open flames.
- Do not use the device near flammable materials.
- Store the silicone parts of the breast pump away from hot surfaces to prevent deformation.
- Always disconnect the device from the power supply after charging if you do not need to charge the device's battery.
- The breast pump should not be left unattended when connected to a power source.
- Never use the device if it has been dropped or damaged in any way.
- Do not use the device if the power cord or plug is damaged.
- Do not connect the device to a power outlet that has been in contact with water or other liquids.
- If the device has been exposed to water or other liquids, do not touch the device directly, disconnect it from the power supply, turn it off and contact the manufacturer.
- Do not connect the device to a power supply higher than specified in the specifications.
- The device is intended for use by a single user. Use of the breast pump by more than one person carries a risk of injury to users and voids the warranty.
- Do not repair or modify the device yourself.
- Use the breast pump only as described in the instructions. Do not use the device for any other purpose.
- Inspect all parts of the breast pump before each use. Do not use the breast pump if any part is clearly damaged.
- Keep the device out of the reach of children and animals.
- Do not clean any parts of the device with abrasive cleaners.
- Do not heat expressed milk in a microwave. Milk heated in a microwave can burn your baby's mouth, as microwave heating can create high-temperature pockets in the liquid. In addition, microwave heating can change the composition of milk.
- Do not allow the milk to boil while heating.
- Be especially careful when using a breast pump or its accessories near children.

02. BEFORE FIRST USE

- Connect the breast pump to the power supply. Charge the batteries for 3 hours.
- Remove the breast pump.
- Disconnect the breast pump parts that will come into direct contact with the breast or milk: remove the membrane and funnel from the milk container, then pull the valve down to disconnect it from the connector.
- Wash and disinfect the parts of the device (refer to the CLEANING AND DISINFECTION section).

CAUTION! The breast pump must be used in an upright position. Using it in any other position (e.g. lying down) will result in the breast pump not fitting properly to the breast and milk spilling out.

03. COMPONENTS

FIGURE A

1. Pumping device
2. Silicone funnel 24 mm
3. Funnel reducers 17 mm / 19 mm / 21 mm
4. Funnel cover
5. Diaphragm x 2
6. Valve x 2
7. Milk container 180 ml
8. Milk container stand
9. USB-C cable

04. DISPLAY ICONS

FIGURE B

1. Suction strength indicator / Battery charge indicator.
2. Suction power reduction button.
3. Suction power increase button.
4. Power button / Device mode change button.
5. Operating mode indicator / Charging indicator.

05. MANUAL PUMP OPERATION

A. SUCTION POWER LEVEL

Each mode has five suction power levels. The number of illuminated LEDs (FIGURE B1) indicates the currently selected suction power level (L1-L5). To decrease or increase the suction power, use the < and > buttons (FIGURES B2-B3).

B. BATTERY CHARGE STATUS

Method 1 – after starting the breast pump, the battery charge indicator (FIGURE B1) will flash 3 times.

Method 2 – hold down the < and > buttons at the same time and the indicator will flash 3 times to show the battery charge status:

- 1 light indicates a battery charge level of 0% -20%,
- 2 lights indicate a battery charge level of 21% -40%,
- 3 lights indicate a battery charge level of 41% -60%,
- 4 lights indicate a battery charge level of 61% -80%,
- 5 lights indicate a battery charge level of 81% -100%.

Connect the device to the charger when the battery is low.

C. TURNING THE DEVICE ON/OFF AND CHANGING THE OPERATING MODE

To turn the device on or off, press and hold the power button (FIGURE B4) for 2 seconds. The device will start operating in stimulation mode. After 2 minutes, the breast pump will automatically switch to deep expression mode. The modes can be changed manually by briefly pressing the power button (FIGURE B4).

D. OPERATING AND CHARGING MODE INDICATOR

The operating mode indicator (FIGURE B5) uses colours to show which mode the device is operating in:

- stimulation – orange,
- deep suction – green,
- mixed – light green.

Battery charging indicator (FIGURE B5) – when the battery is charging, the indicator flashes green. When the device is fully charged, the indicator lights up green continuously.

06. APPLICATION OPERATION

The breast pump is equipped with Bluetooth connectivity, which allows you to connect to a dedicated mobile app. This gives you more control over milk expression and access to useful monitoring features (FIGURE E).

A. INSTALLING THE APP AND PAIRING WITH THE DEVICE

- Download the NENO app from the App Store (iOS) or Google Play (Android).
- Install the app on your smartphone.
- Make sure Bluetooth is enabled on your phone.
- Turn on the breast pump.
- Open the app and follow the instructions on the screen.
- The app will automatically detect the device. Select your breast pump from the list of available devices and accept the connection.
- Once connected, the device will be synchronised with the app.

B. MAIN FEATURES OF THE APP

- **Remote control:** Control the device directly from your phone – turn the breast pump on and off, change operating modes and adjust the suction strength in real time, without having to touch the device itself.
- **Session recording:** The ability to accurately record data related to each pumping session is a convenient and practical solution that allows you to track progress, analyse milk production and better tailor your schedule to your individual needs. All data is automatically organised, making it easy to monitor and plan.
- **History and statistics:** The app stores the history of all sessions – with date, time, duration and amount of milk. The data is presented in clear graphs, which facilitates analysis and feeding planning. You can view the data on a daily, weekly or monthly basis.

C. BENEFITS OF RECORDING DATA

- **Better planning** – you can see when and how much milk you have expressed, which helps you adjust your pumping rhythm to your schedule.
- **Support in building lactation** – it is easier to assess which times of day are most effective and how your body responds to changes.
- **Monitoring changes** – thanks to the history, you can notice any decreases or increases in milk supply and react faster.

07. CLEANING AND DISINFECTION

Cleaning and disinfection are two different activities. They must be done separately to protect your baby and ensure the breast pump works properly.

Washing – rinsing the surface of the device and removing any dirt.

Disinfection – killing organisms such as bacteria or viruses that may be present on the surface of the device.

Funnel, membrane, valve, reducers, milk container:

Before first use – wash and disinfect.

After each use – wash.

Once a day – disinfection.

A. CLEANING:

- Disassemble all parts of the breast pump (except for the pumping device) and wash them in warm soapy water, then rinse. These steps should be performed immediately after using the device. This will remove milk residue and prevent the growth of bacteria.
- Only use tap water that is safe to drink for washing. Use bottled water if the water from your tap does not meet this requirement.
- Do not use brushes to clean silicone parts. This may cause damage.

- Use a mild dishwashing liquid or soap.
- Allow the parts to dry naturally. Store dry parts of the device in a cool, dry place when not in use.

B. DISINFECTION:

Membrane, valve, reducers, funnel, milk container:

- Fill a pot or steriliser with water so that all parts (membrane, valve, reducers, funnel and milk container) are covered when placed inside. Bring to the boil. Bring the water to a boil. Turn off the heat and place the breast pump parts in the hot water for 10 minutes. **Do not keep boiling during disinfection, as this may damage the parts.**
- Allow the parts to dry naturally. Store the dry parts of the set in a cool, dry place when not in use.

C. CLEANING THE PUMP:

- Wipe the pump unit with a soft, damp cloth and mild dishwashing liquid or soap. Other cleaning agents or abrasive materials may damage the device.
- When not in use, store the pump unit in a cool, dry place.

08. ASSEMBLY OF THE LACTATOR

FIGURE C

Attach the parts as shown in FIGURE C1-C4.

NOTE: Wash your hands thoroughly with soap and water before touching the breast pump and your breasts. Before each use, make sure that all parts of the set are clean and dry.

- Connect the valve to the milk container in the direction shown in the diagram (FIGURE C1).
- Attach the funnel to the milk container, starting by placing the connector in the groove. Ensure that the funnel is in the correct position (FIGURE C2).
- Place the membrane in the milk container (FIGURE C3). Pay particular attention to ensuring that the membrane tabs are properly aligned with the holes in the milk container.
- Attach the pumping device to the top of the milk container by pressing down on the membrane (FIGURE C4). When the device is correctly attached, you will hear a distinctive click.

Tips:

- Before each use, check that the breast pump parts are not damaged. Replace any damaged parts with new ones if necessary.
- Always check that all parts are clean before using the device.
- If the diameter of the funnel is too large, use the included reducer by inserting it into the centre of the funnel.

09. USING THE BREAST PUMP

FIGURE D

- Place the funnel on your breast so that the nipple is inside the drainage channel. Press the funnel and the breast pump connection to your breast using your thumb and index finger.
- Place the breast pump in your nursing bra/girdle. Turn on the breast pump by pressing the power button and select the mode that suits you.

NOTE: The breast pump starts in stimulation mode. Set the suction strength to a level that is comfortable for you using the and >img alt="plus icon" data-bbox="245 745 260 760"/> buttons or the app.

- Stimulation mode - rapid pumping that mimics a baby's natural suckling, stimulating milk flow. After 2 minutes, the breast pump will automatically switch to deep expression mode. Adjust the suction strength using the and >img alt="plus icon" data-bbox="245 785 260 800"/> buttons or the app to find the highest suction strength that is comfortable for you. If milk starts flowing from your breast before two minutes have passed, press the mode button to manually switch to deep expression mode. You can use the app for this.
- Deep expression mode - slower pumping that gently and efficiently extracts milk from the breast. In milk expression mode, increase the suction strength until you feel slight discomfort (not pain), and then reduce the suction strength by one level.
- Mixed mode - mimics a baby's sucking reflex. Can be used after expressing milk to help empty the breast and increase lactation.

10. MILK TRANSFER

- Place the breast pump on the stand. Gently remove the pump by lifting it upwards. Then remove the membrane.
- Hold the milk container horizontally with the funnel facing upwards to prevent spillage.
- Carefully pour the milk into a bottle or storage container.

After use, remove and clean all parts of the breast pump.

11. CHARGING THE BREAST PUMP

Fully charge the breast pump battery before storing it if you do not intend to use the device for a long period of time. Charge the battery before it is completely depleted. This will extend the battery life. If the breast pump has been stored in a hot place, the battery may not work at first. Allow the breast pump to cool down for an hour before use.

You can check the battery charge level in the app or using the methods described in the MANUAL PUMP OPERATION section.

While the battery is charging, the charging indicator (FIGURE B5) flashes green. When the device is fully charged, the indicator lights up green.

12. REPLACEMENT OF CONSUMABLE PARTS

Consumable parts are subject to natural wear and tear during use of the breast pump and are therefore not covered by the manufacturer's warranty.

Worn parts of the breast pump that come into contact with milk must be replaced. A worn part has a crack, deformation, dullness or a noticeable change in colour.

- The membrane should be replaced every 3-4 months (if the breast pump is used 1-3 times a day) or every 6-8 weeks (if the breast pump is used more than 3 times a day).
- The valve should be replaced every 3-4 months (if the breast pump is used 1-3 times a day) or every 6-8 weeks (if the breast pump is used more than 3 times a day).
- The funnel should be replaced every 6 months if the breast pump is used more than 3 times a day.

13. LONG-TERM STORAGE

The breast pump has a built-in battery. To prevent damage, the device enters storage mode after a long period of inactivity. To turn off storage mode, connect the device to a power source and charge it for 3 hours. Before storing the breast pump for an extended period of time, make sure that:

- All parts of the breast pump are clean and dry.
- You have cleaned the pumping device.
- The device battery is fully charged.

14. FAQ

The device does not start: make sure that the breast pump is connected to a working power outlet. If the breast pump has no power source, its display will not turn on.

No/weak suction: make sure that all parts of the breast pump are positioned correctly. Disassemble the breast pump and make sure that all parts are clean and undamaged. Reassemble the breast pump according to the instructions in the ASSEMBLING THE BREAST PUMP section. During pumping, make sure that the entire surface of the funnel is in contact with the breast.

If more milk is expressed than the container can hold, stop pumping and carefully pour the excess into another container, then use a damp (not wet) cloth to wipe the breast pump. Wash the parts of the breast pump that contain milk according to the instructions in the CLEANING AND DISINFECTION section. Once all parts of the device are dry, reassemble them.

15. SPECIFICATIONS

Milk container volume: 180 ml

Battery capacity: 3.7V 1200mAh Li-ion

Input: 100-240V, 50-60Hz

Output: 5V 1A

Operating time: up to 180 min

Charging time: approx. 180 min

Pump dimensions: 7.5 x 7.5 x 3.7 cm

Breast pump weight: 112 g

Set weight: 200 g

Suction strength:

1. Stimulation mode (5 levels): 10-29 kPa
 2. Deep expression mode (5 levels): 16-36 kPa
 3. Mixed mode (5 levels): 10-29 kPa
- Permissible difference ± 5 kPa.

16. WARRANTY CARD / COMPLAINTS

The product is covered by a 24-month warranty. The warranty terms and conditions can be found at:

<https://neno.pl/gwarancja>

Details, contact information and service address can be found at: <https://neno.pl/kontakt>

Specifications and kit contents are subject to change without notice. We apologise for any inconvenience.

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EN

The crossed-out wheeled bin symbol indicates that unwanted electrical or electronic equipment, including accessories (such as power supplies or cables) and components (e.g., batteries, if included), must not be disposed of with household waste. Instead, these items should be taken to an appropriate collection point where they will be accepted free of charge for proper disposal or recycling. This requirement is in accordance with the revised WEEE Directive (2012/19/EU) and Regulation (EU) 2023/1542, which amends Directive 2008/98/EC and Regulation (EU) 2019/1020, and repeals Directive 2006/66/EC. Proper disposal of such devices helps prevent environmental damage. Information on local collection points is available from the relevant municipal authorities. Improper disposal of waste may be subject to penalties under applicable local laws.

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