nenob

Belisa Twin

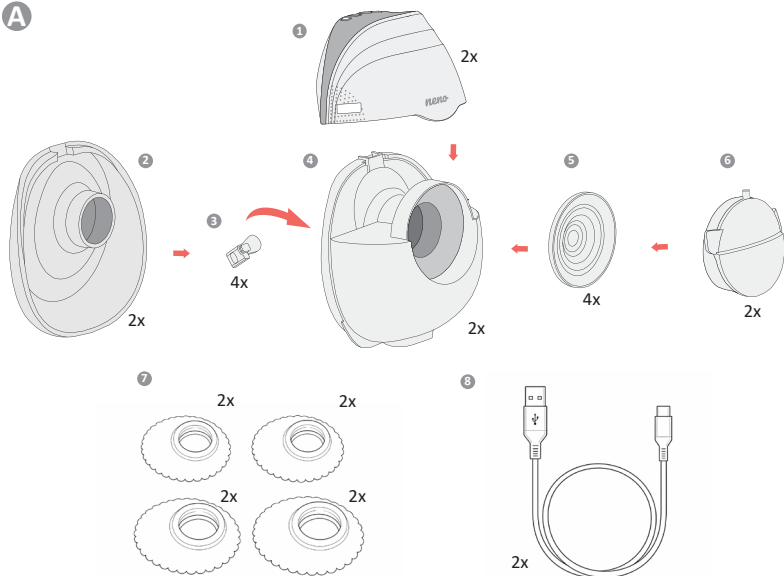
Table of contents


Spis treści

6	Instrukcja obsługi	PL
9	User manual	EN
13	Bedienungsanleitung	DE
16	Návod k použití	CZ
19	Használati utasítás	HU
23	Používateľská príručka	SK
26	Användarhandbok	SE
29	Käyttäjän käsikirja	FI
32	Brukerhåndbok	NO
35	Brugermanual	DK
39	Gebruikershandleiding	NL
42	Manual del usuario	ES
46	Manuale utente	IT
49	Manuel de l'utilisateur	FR
52	Manual de utilizare	RO
56	Korisnički priručnik	HR/BA
59	Uputstvo za upotrebu	RS/ME
62	Uporabniški pravilnik	SI
65	Εγχειρίδιο χρήστη	GR
69	Lietotāja rokasgrāmata	LV
72	Vartotojo vadovas	LT
75	Kasutusjuhend	ET
78	Посібник користувача	UA
82	Ръководство за потребителя	BG
86	Упатство за употреба	MK
89	Manual do utilizador	PT


A


B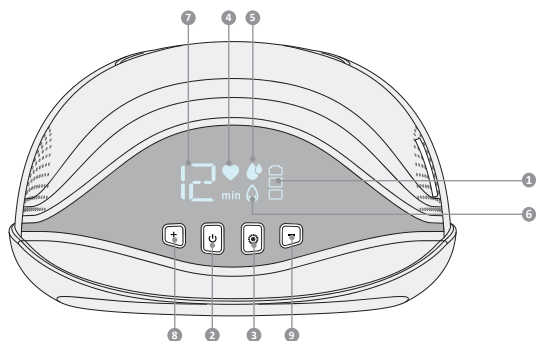


C






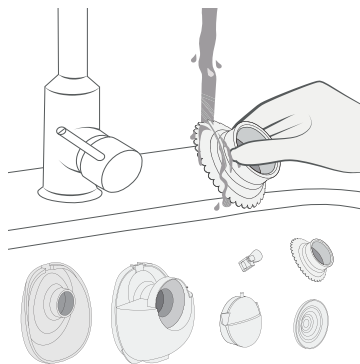
D




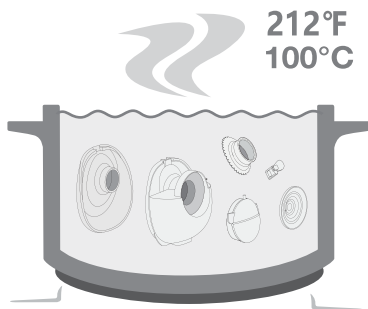
E

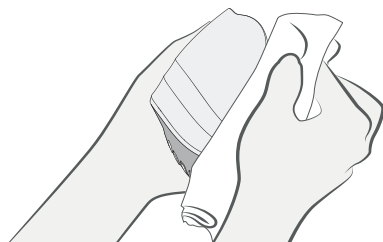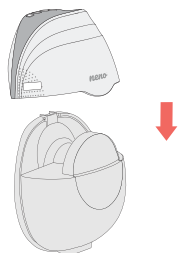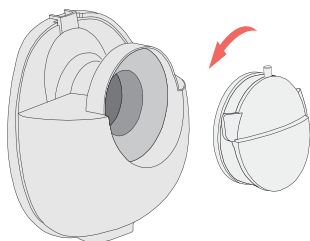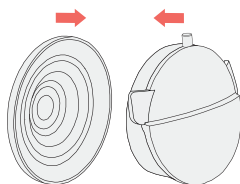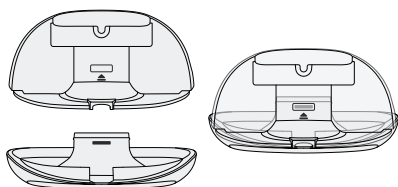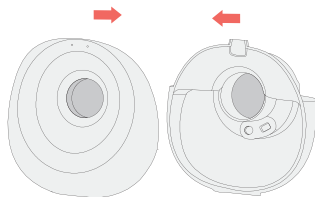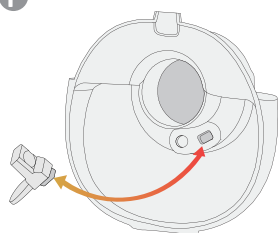F

G

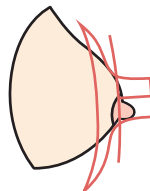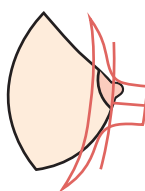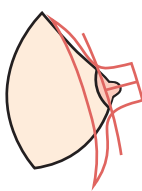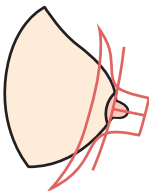PL

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Szanowny Kliencie,

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USER MANUAL

Dear Customer,

The Neno Belisa Twin device you have purchased consists of two electronic, three-phase shell-type breast pumps designed for expressing milk from a woman's breasts after childbirth. They have been designed with comfort, effectiveness and discretion in mind. This user manual contains all the information necessary for correct use. Please read it carefully and keep it for future reference.

01. PRECAUTIONS

- Do not use the breast pump during pregnancy – this may cause premature labour.
- Do not use the device if you suffer from chronic illnesses or hypersensitivity, as using the breast pump may cause you pain. If using the device causes pain or severe discomfort, switch it off and remove the breast pump from your breast.
- Do not place the product in areas where it risks falling into a sink, bath or other water-filled containers.
- Do not use the device in the bath or in the shower.
- Do not place the device under water or in other liquids.
- Do not wash the device under running water.
- The breast pump and its components are not heat-resistant – keep the device away from hot surfaces and open flames.
- Do not use the device near flammable materials.

- Store the silicone parts of the breast pump away from hot surfaces to prevent them from warping.
- Always unplug the device from the mains after charging, unless you need to charge the device's battery.
- The breast pump should not be left unattended whilst connected to a power source.
- Never use the device if it has been dropped or damaged in any other way.
- Do not use the device if the power cord or plug is damaged.
- Do not plug the device into a power socket that has been in contact with water or other liquids.
- If the device has been exposed to water or other liquids – do not touch the device directly, disconnect it from the power supply, switch it off and contact the manufacturer.
- Do not connect the device for charging to a voltage higher than that specified in the technical specifications.
- The device is intended for use by a single user. Use of the breast pump by more than one person carries a risk of harm to users' health and will invalidate the warranty.
- Do not attempt to repair or modify the device yourself.
- Use the breast pump only as described in the instructions. Do not use the device for any other purpose.
- Inspect all parts of the breast pump before each use. Do not use the breast pump if any part of it is damaged.
- Keep the device out of the reach of children and pets.
- Do not clean parts of the device with abrasive cleaners.
- Do not heat expressed milk in a microwave. Milk heated in a microwave may burn your baby's mouth, as heating with microwave waves can create pockets of high temperature within the liquid. Additionally, microwave waves may alter the composition of the milk.
- Do not allow the milk to boil whilst heating it.
- Take particular care when using the breast pump or its accessories near children.

02. BEFORE FIRST USE

- Connect the breast pump to the mains. Charge the batteries for 2.5–3 hours.
- Remove the breast pump unit.
- Detach the parts of the breast pump that will come into direct contact with the breast or milk: remove the funnel from the milk container, then remove the valve. Remove the air chamber cover and the membrane from the milk container.
- Wash and sterilise the parts of the device (refer to the CLEANING AND STERILISATION section).

IMPORTANT! The breast pump must be used in an upright position. Using it in any other position (e.g. lying down) will result in the breast pump not fitting the breast properly and milk spilling.

03. COMPONENTS

FIGURE A

1. Pump unit x 2
2. 27 mm silicone funnel x 2
3. Valve x 4
4. 180 ml milk container x 2
5. Diaphragm x 4
6. Air chamber lid x 2
7. 24/21/18/15 mm funnel adapters x 2
8. USB-C cable x2

04. DISPLAY ICONS

FIGURE B

1. Battery charge level - The battery icon shows the battery charge level. Connect the device to a charger when the battery is running low.
2. Power/Pause button – hold the button for 5 seconds to switch the device on or off. When the device is switched on, press briefly to pause/ resume the breast pump.
3. Mode button – press the button to change the device's operating mode. There are 3 operating modes available (stimulation, deep expression, mixed).
4. Stimulation mode icon
5. Deep expression mode icon
6. Combined mode icon
7. Suction strength/session duration indicator – shows the currently selected suction strength level/operating time
8. Suction strength increase button – press to increase the suction strength.
9. Suction strength reduction button – press to reduce suction strength.

05. CLEANING AND DISINFECTION

Cleaning and disinfection are two separate processes. They must be carried out separately to protect your baby and ensure the breast pump works correctly.

Cleaning – washing the surface of the device and removing dirt.

Disinfection – killing organisms such as bacteria or viruses that may be present on the surface of the device.

Funnel, membrane, valve, reducers, milk container, air chamber lid:

Before first use – wash and disinfect.

After each use – cleaning.

Once a day – disinfection.

A. CLEANING (FIGURE C):

- Dismantle all parts of the breast pump (except the pump unit) and wash them in warm soapy water, then rinse. These steps should be carried out immediately after using the device. This will remove milk residue and prevent the growth of bacteria.
- Use only tap water that is safe to drink for cleaning. Use bottled water if your tap water does not meet this requirement.
- Do not use brushes when cleaning silicone parts. This may cause damage.
- Use a mild washing-up liquid or soap.
- Allow the parts to air dry. Store the dry parts of the device in a cool, dry place when not in use.

B. DISINFECTION (FIGURE D):

Diaphragm, valve, reducers, funnel, milk container, air chamber lid:

- Fill a saucepan or steriliser with water so that all the parts placed inside (membrane, valve, pressure regulators, funnel, milk container and air chamber lid) are covered once placed inside. Place the breast pump parts in **boiling water** for 10 minutes. **Do not continue boiling them** – this may damage the parts.
- Allow the parts to dry naturally. Store the dry parts of the set in a cool, dry place when not in use.

C. CLEANING THE PUMPING UNIT (FIGURE E):

- Wipe the pump unit with a soft, damp cloth and mild washing-up liquid or soap. Other cleaning products or abrasive materials may damage the unit.
- When not in use, store the pump unit in a cool, dry place.

06. ASSEMBLY OF THE BREAST PUMP

FIGURE F

Assemble the parts as shown in **FIGURES F1–F6**.

NOTE: Wash your hands thoroughly with soap and water before touching the breast pump and your breast. Before each use, ensure that all parts of the set are clean and dry.

- Place the valve in the correct opening of the milk container as shown in **FIGURE F1**.
- Attach the funnel to the milk container, starting with the opening at the top. Ensure that the funnel is in the correct position (**FIGURES F2–F3**).
- Place the membrane in the air chamber lid (**FIGURE F4**). Press it down with your fingers and check that all edges fit snugly.
- Fit the chamber lid with the membrane into the milk container. (**FIGURE F5**).
- Attach the pump unit to the top of the milk container (**FIGURE F6**). When the unit is correctly attached, you will hear a distinctive click.

Tips:

- Before each use, check that the parts of the breast pump are not damaged. If necessary, replace damaged parts with new ones.
- Always check that the parts are clean before using the device.
- If the funnel's diameter is too large, use the included reducer by inserting it into the centre of the funnel.

07. USING THE BREAST PUMP

- Place the funnel on your breast so that the nipple is inside the funnel tunnel (**FIGURE G**). Press the funnel and the breast pump connector against your breast using your thumb and index finger.
- Place the breast pump in your bra or nursing bra. Switch on the breast pump by pressing the power button and select the operating mode that suits you.

NOTE: The breast pump starts in stimulation mode. Set the suction strength to a level that is comfortable for you using the +/- buttons.

- Stimulation mode – rapid pumping that mimics a baby's natural suckling at the breast, stimulating milk flow. After 2 minutes, the breast pump will automatically switch to deep expression mode. Adjust the suction strength using the +/- buttons to find the highest suction level that is comfortable for you. If milk starts to flow from your breast before two minutes have elapsed, press the mode change button to manually switch to deep expression mode.
- Deep expression mode – slower pumping that gently and efficiently draws milk from the breast. In expression mode, increase the suction strength until you feel slight discomfort (not pain) and then reduce the suction strength by one level.
- Combined mode – mimics a baby's suckling reflex. It can be used after expressing milk to help empty the breast and boost milk production.

08. EMPTYING THE MILK CONTAINER

- Gently remove the pump by lifting it upwards.
- Hold the milk container horizontally, with the spout facing upwards, to prevent spillage.
- Carefully pour the milk into a bottle or storage container.

After use, remove and clean all parts of the breast pump.

09. CHARGING THE BREAST PUMP

Fully charge the breast pump's battery before storing it if you do not intend to use the device for a long period of time. Charge the device's

battery before it is completely drained. This extends the battery's lifespan. If the breast pump has been stored in a hot place, the device's battery may not work at first. Allow the breast pump to cool down for an hour before use.

10. REPLACEMENT OF CONSUMABLE PARTS

Consumable parts are subject to natural wear and tear during the operation of the breast pump and are therefore not covered by the manufacturer's warranty.

Worn parts of the breast pump that come into contact with milk must be replaced. A worn part will show cracks, deformation, dullness or a noticeable change in colour.

- The membrane should be replaced every 3–4 months (if the breast pump is used 1–3 times a day) or every 6–8 weeks (if the breast pump is used more than 3 times a day).
- The valve should be replaced every 3–4 months (if you use the breast pump 1–3 times a day) or every 6–8 weeks (if you use the breast pump more than 3 times a day).
- The funnel, milk container and air chamber lid should be replaced every 6 months if you use the breast pump more than 3 times a day.

11. LONG-TERM STORAGE

The breast pump has a built-in battery. To prevent damage, the device enters storage mode after a long period of inactivity. To exit storage mode, connect the device to a power source and charge it for 3 hours. Before storing the breast pump for an extended period, ensure that:

- All parts of the breast pump are clean and dry.
- You have cleaned the pumping unit.
- The device's battery is fully charged.

12. FAQ

The device does not start: ensure that breast pump is charged or connected to a working power socket. If the breast pump has no power source, its display will not turn on.

No / weak suction: make sure all parts of the breast pump are fitted correctly. Take the breast pump apart and ensure all parts are clean and undamaged. Reassemble the breast pump following the instructions in the ASSEMBLY section. While pumping, ensure that the entire surface of the funnel is in contact with your breast.

More milk has flowed from the breast than the container can hold: Use a damp (not wet) cloth to wipe the breast pump. Wash the parts of the breast pump that contain milk in accordance with the instructions in the CLEANING AND DISINFECTION section. Once all parts of the device are dry, reassemble them.

13. SPECIFICATIONS

Milk container capacity: 180 ml x 2

Milk container material: PP

Battery: 3.7V 1000 mAh Li-ion

Configuration: 2 x 500 mAh cells connected in parallel

Input: 5V 2A

Charging time: approx. 180 mins

Operating time: approx. 210 mins

Dimensions of one breast pump: 12.8 x 11.1 x 6.7 cm

Weight of one breast pump: 235 g

Suction strength:

Stimulation mode (12 levels): 6–24 kPa

Deep expression mode (12 levels): 12–36 kPa

Combination mode (12 levels): 12–36 kPa

Permissible deviation ± 5 kPa.

14. WARRANTY CARD / COMPLAINTS

The product is covered by a 24-month warranty. The warranty terms and conditions can be found at: <https://neno.pl/gwarancja>

Details, contact information and the service address can be found at: <https://neno.pl/kontakt>

Specifications and contents of the set are subject to change without notice. We apologise for any inconvenience.

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